# Istruzioni d'uso

ver. 3 - 23/09/2021

Biologia Molecolare

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## PAI-1 (4G/5G) REAL TIME (FRET)

**REF**

AA1034

**CE**

25 TEST

**CND**

W0103020506

**IVD**

**NUCLEAR LASER MEDICINE S.r.l.**

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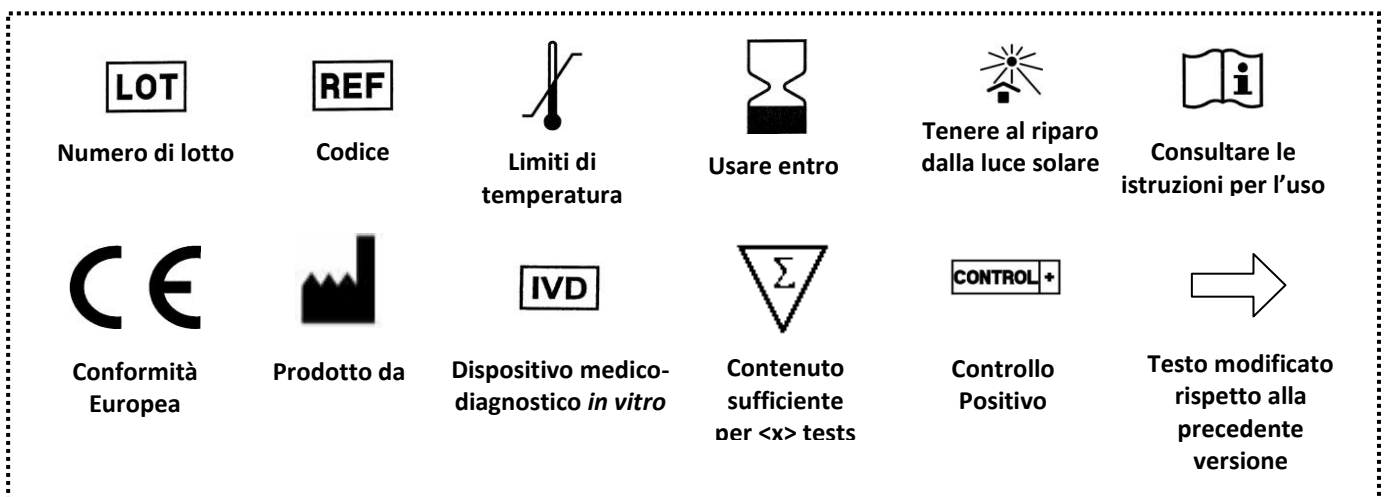
**IVD**

**NUCLEAR LASER MEDICINE S.r.l.**

HEADQUARTER: Via Cascina Conighetto – BUSINESS OFFICES: Viale delle Industrie, 3 - 20049 SETTALA (MI), ITALY

Phone: (+39) 02/95.24.51 - Fax (+39) 02/95.24.52.37 - (+39) 02/95.24.52.38 WEB: [www.nlm.it](http://www.nlm.it) - E-MAIL: [segreteria@nlm.it](mailto:segreteria@nlm.it)

ISO 9001 Quality Management System Organization certified and ISO 13485 Medical Sector Quality Management System certified (IMQ certification body - CSQ certification).

## INTENDED USE

AA1034 kit provides reagents for the genotyping of Plasminogen activator inhibitor-1 (PAI-1) sequence gene, which includes polymorphisms of type insertion/deletion of one G (4G/5G). This polymorphism is in the promoter region of gene and is involved in arterial and deep venous thrombosis through the amplification of target sequences and melting curve analysis.

Amplification enzyme assigned: **green cap** NLM code BA087, followed by size indication, which could change depending on enzyme quantity supplied (i.e.: BA087/60, BA087/100, etc).

This device has been validated with:

- a) manual column-based extraction (NLM code AA1001) and Rapid Extraction (NLM code AA898 only for RotorGene) and with the most common automatic systems for nucleic acids purification.
- b) Real time PCR Rotorgene (Corbett Research/Qiagen) and CFX (BioRad) instruments.

## INTRODUCTION

Venous thrombosis is one of the most common cardiovascular diseases; it occurs with a frequency that reaches 1/1000 case/year.

Thrombotic risk is due both to environmental factor (age, pregnancy, oral contraceptive etc.) and genetic predisposition.

One of the factors involved in the evaluation of thrombotic risk is plasminogen activator inhibitor-1 (PAI-1); his main role is the dissolution of clots for inhibition of the conversion of plasminogen in plasmin which has fibrinolytic activity. The fibrinolysis is the physiological process that has in its terminal event the dissolution of the grid of fibrin formed during coagulation. Fibrinolysis keeps integrity of the clot for the time necessary to stop haemorrhage and prevents the formation of intravascular thrombosis, due to the spontaneous activation of the coagulation pathway.

In the promoter region of the gene coding for this protein, there is an insertion/deletion (polymorphism 4G/5G), which is associated with different levels of plasminogen in the plasma and therefore with thrombotic risk.

## REFERENCES

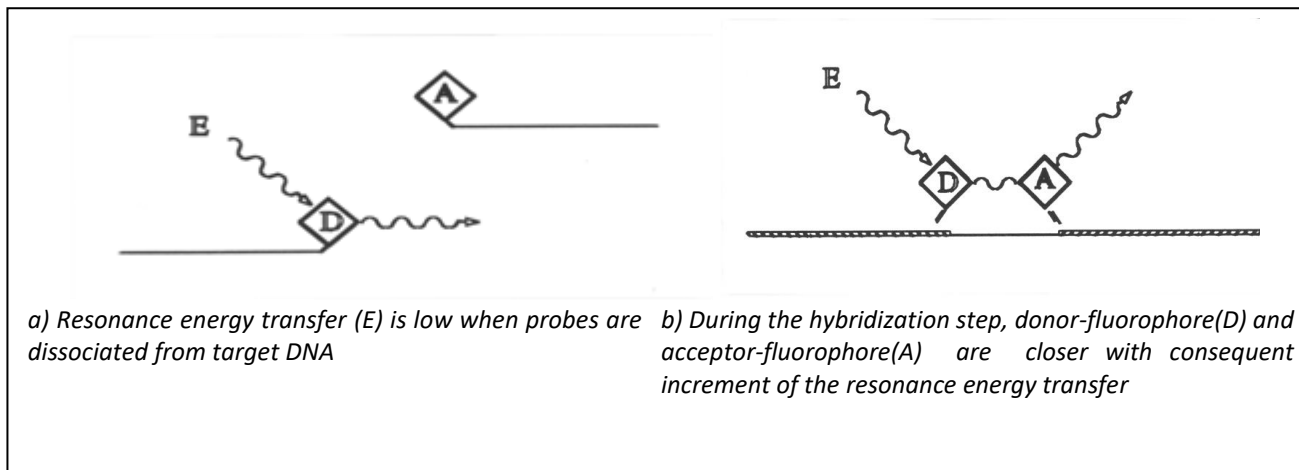
- *Filippo Aucella et al. Nephrol Dial Transplant (2003) 18: 1142–1146*
- *M.D.I. Vergouwen et al. Stroke. 2004;35:1280-1283*
- *Markus Nauck et al. Clinical Chemistry 45:8 1141–1147 (1999)*
- *R.J. Pegoraro et al. Cardiovascular Journal Of South Africa Vol 16, No. 5, Sept.r/Oc.r 2005*
- *Argiros E. Tsantes et al. Thromb Haemost 2007; 97: 907–913*
- *T.Buchholz et al. Human Reproduction Vol.18, No.11 pp. 2473±2477, 2003*
- *Per-Gunnar Wiklund et al. Stroke. 2005;36:1661-1665*

## PRINCIPLE OF THE PROCEDURE

The test is based on the Real Time PCR, a technology that allows monitoring in real time of the amplification process. Hybridization of specific fluorescent probes to the target DNA results in fluorescence resonance energy transfer (FRET) between two fluorophores.

Extracted DNA is amplified in the presence of two contiguous probes that are both complementary to the sequence which contains the mutation: probes at the 5' is marked by a donor-fluorophore, probe in 3' carries the acceptor-fluorophore. During the PCR, probes hybridizes to the target DNA and fluorophores are very close to each other; in presence of a source of light there is an energy transfer from the donor to the acceptor fluorophore which emits a signal (of a specific wavelength); this allows monitoring of the PCR process. Emitted signals are proportional to the accumulation of PCR product. Without a target DNA no energy transfer is possible.

Hybridization probes allow identification of mutations in the target sequence through the melting curve analysis; after amplification PCR product is submitted to a raising temperature with consequent detachment of the probes from the DNA. If the probe is exactly complementary to the wild-type sequence, wild-type templates will form stronger hybrids with the probe and will consequently melt at a higher temperature than templates that contain a mutation. The different melting temperature of the two possible hybrids allows identification of each unknown sample as mutant, wild type, or heterozygote (one allele of mutant and one allele of wild-type sequence).

## PRODUCT COMPOSITION

Store at -25/-15°C

| Reagents          | No. of vials  | Volume/vial | Caution   |
|-------------------|---------------|-------------|---|
| Mix 1             | 5 for 5 tests | 100 µl      |   |
| MIX 2 PAI-1 4G/5G | 5 for 5 tests | 65 µl       |  |

⇒ The amount of reagents indicated in the table is referred to the standard size of the kit. Reduced packaging is available on request for evaluation and/or demonstration of the product.

## STABILITY AND STORAGE

- All the reagents are stable up to the expiry date indicated on the label when stored at the correct temperature (-25/-15°C).
- Thaw Amplification Mix on ice or at +2/+8°C
- Specific probes in the Mix 2 are photosensitive: avoid direct and prolonged exposure to light
- **Master Mix** have to be used immediately after preparation; once dispensed in PCR tubes, the remaining reagents have to be discarded. Avoid prolonged exposure to light.
- Keep amplification reagents on ice during the entire procedure.
- At the end of each assay store reagents back at their correct storage temperature.

## PRECAUTIONS

- Only professional and opportunely trained personnel should use this kit. Handle this product according to established Good Laboratory Practices and universal precautions; wear personal protective apparel.
- When DNA samples or amplification products are handled, use aerosol-resistant pipette tips to avoid pipettes contamination.
- Discard used materials as bio hazardous waste.
- Do not eat, drink, smoke or apply cosmetics in areas where reagents or specimens are handled.
- If skin or mucous membrane exposure occurs, immediately wash the area with copious amount of water. Seek medical advice.
- Do not use components beyond the expiration date indicated on the kit box
- Do not mix reagents from different lots.
- Reagents have to be preserved separated from possible contaminants (as DNA samples and amplification products).
- Avoid repeated freezing and thawing of the mix
- It is recommended to make analysis in three different areas:
  - Area 1: pre-PCR (samples handling and extraction)
  - Area 2: Master Mix preparation.
  - Area 3: post-PCR (Real Time PCR)
- Don't use the device if the box is damaged; contact the supplier.

It is advisable to have constant and uniform laboratory temperature, avoid to place the instruments near heating/cooling sources that may compromise the correct work

## MATERIALS REQUIRED BUT NOT SUPPLIED

### ZONE 1

DNA Extraction Kit  
Vertical downflow airbox  
Dedicated adjustable volume pipettes set  
Aerosol-resistant pipettes tips and disposable tubes

### ZONE 2

Vertical downflow airbox  
Dedicated adjustable volume pipettes set  
Aerosol-resistant pipettes tips  
0,2 ml PCR tubes, strips or plates  
DNA polymerase with green cap

### ZONE 3

Real Time PCR instrument

## PROCEDURE

### DNA EXTRACTION

Manual purification: NLM code AA1001, NLM code AA898 (Only for RotorGene).  
For extraction with automatic systems follow the specific manufacturer's IFUs.

Use only EDTA or Citrate as anticoagulants, not heparin. Use fresh or correctly stored (+2/+8°C for up to 2 days or -25/-15°C for longer periods) blood samples.

### AMPLIFICATION

#### Warning:

- Avoid using blood sample after repeated freeze/thaw cycles or not correctly stored in order to avoid bad assay results.
- Use dedicated PCR tubes only
- We recommend to set the thermal profile before the amplification mix preparation.
- ⇒ • The Positive Control is for single use only. Discard the remaining material in the tube.

#### Prepare the master mix as follows:

| Reagents                          | Volume/sample |
|-----------------------------------|---------------|
| Mix 1                             | 10,75 µl      |
| Mix 2                             | 10 µl         |
| DNA polymerase (5U/µl), green cap | 0,25 µl       |

- Prepare the Master Mix for the number of samples extracted + 1 more volume (if "n" ≤ 10) or + 2 more volumes (if "n" >10).
- Mix gently and dispense **21 µl** of Master Mix in the previously marked 0,2 ml test tubes.
- Add **4 µl** of extracted DNA to each tube and mix pipetting up and down (**DNA concentration to be used in this test should be 30-40 ng/µl**).

## THERMAL PROFILE SETTING AND ANALISYS

### ROTORGENE

- Use the software function “Edit Samples” in order to insert samples positions.
- In the “Setting” section set the “reaction volume” to 25 µl and in “machine option” select the used rotor (36/72 position). Check in the “Channels” window if **fret 1** (Source 470 nm, Detector 610 hp) is already present, otherwise click on “Create new” in order to insert it.
- In the menu “ View → gain calibration” set autocalibration temperature to 54°C on channel fret 1 (**Min reading 15 FI; Max reading 20 FI**) and activate the checkbox “Perform calibration before first acquisition”.
- In the “Profile” section set “thermal-profile” specific for your instrument (ref. tables on the next page)
- In “melt” step, keep selected 1 degree in temperature increasing parameter: **do not change the pre-selected parameter!**
- Place tubes in the RotorGene and close the lid.

**Warning:** in case of assay(s) with multiple coagulation factors, place in first position MTHFR C677T samples and then the other factors.

- Run the assay by clicking the “Start” button .

### RotorGene 3000

| Cycle  | Cycle Point   |
|--|---|
| Denature: 95°C, 2 min  |   |
| Cycling (40 repeats)   | Step 1: 95°C, 15 sec                                  |
|  | Step 2: 54°C, 20 sec; acquiring to Cycling A (fret 1) |
|  | Step 3: 72°C, 10 sec                                  |
| Hold: 25°C, 10 min   |   |
| Hold 2: 95°C, 1 min  |   |
| Hold 3: 40°C, 1 min  |   |
| Melt: (40-80°C), hold 15 sec on 1 <sup>st</sup> step, hold 5 sec on next steps, acquiring to Melt A (fret 1) |   |

### RotorGene 6000 or Q

| Cycle   | Cycle Point   |
|---|---|
| Denature: 95°C, 2 min   |   |
| Cycling (40 repeats)  | Step 1: 95°C, 15 sec                                  |
|   | Step 2: 54°C, 20 sec; acquiring to Cycling A (fret 1) |
|   | Step 3: 72°C, 10 sec                                  |
| Hold: 25°C, 10 min  |   |
| Hold 2: 95°C, 1 min   |   |
| *Melt: (40-80°C), hold 90 sec on 1 <sup>st</sup> step, hold 5 sec on next steps, acquiring to Melt A (fret 1) |   |

\*Select the option “Gain optimisation” and enter “90” as maximum fluorescence value.

## INTERPRETATION OF RESULTS

Click “Analysis” and select the “Melt” option: a window with the graphic analysis of the assay will be displayed.

The graph shows the melting-curve of each sample(s) that could be:

- a single peak at  $57,5 \pm 2,0^{\circ}\text{C}$  for homozygous 4G samples
- a single peak at  $64 \pm 2,0^{\circ}\text{C}$  for homozygous 5G samples
- a double peak for heterozygous samples

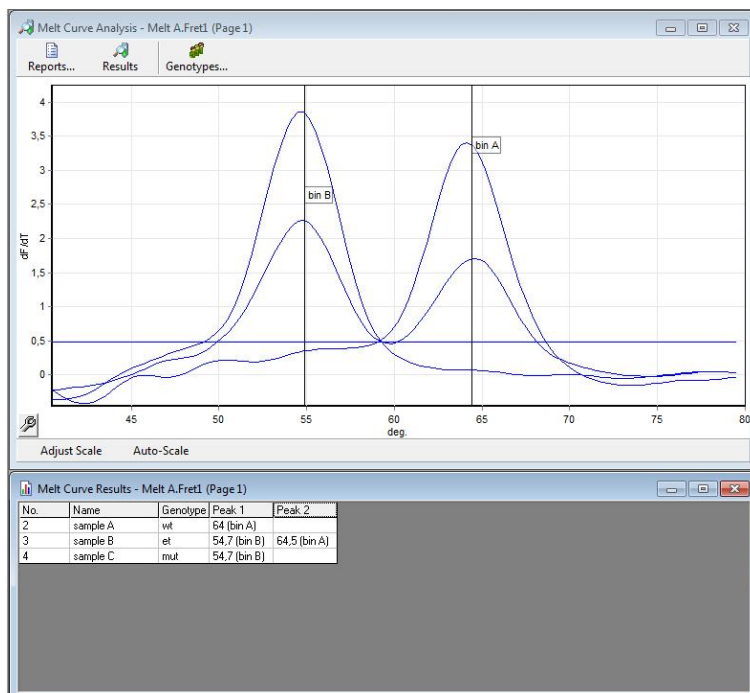
**Warning:** do not consider peaks out of the temperature range

In order to obtain the typing samples select “New bin”: a cursor will appear on the graph that has to be placed in correspondence of the peak 4G (Bin A); set the width in cycles at 4; repeat the same operation for the peak 5G (Bin B).

Click the “Genotypes” icon; a new window will be displayed. Set item as follow: homozygous 4G samples (Bin A), heterozygous samples (Bin A, Bin B), homozygous 5G samples (Bin B).

Click “Report” in order to obtain the final result.

⇨ GENERIC example of Melting curves analysis performed on three different genotypes

## WARNING

- If there is a mismatch between the sample typing and the peaks on the graph, the removal of background signals is required. Click on the "Threshold" window: a cursor will appear on the graph that allows the adjustment of the threshold value.

Example:

fig. 1 (threshold = 0)  $\Rightarrow$  incorrect typing (heterozygous sample)


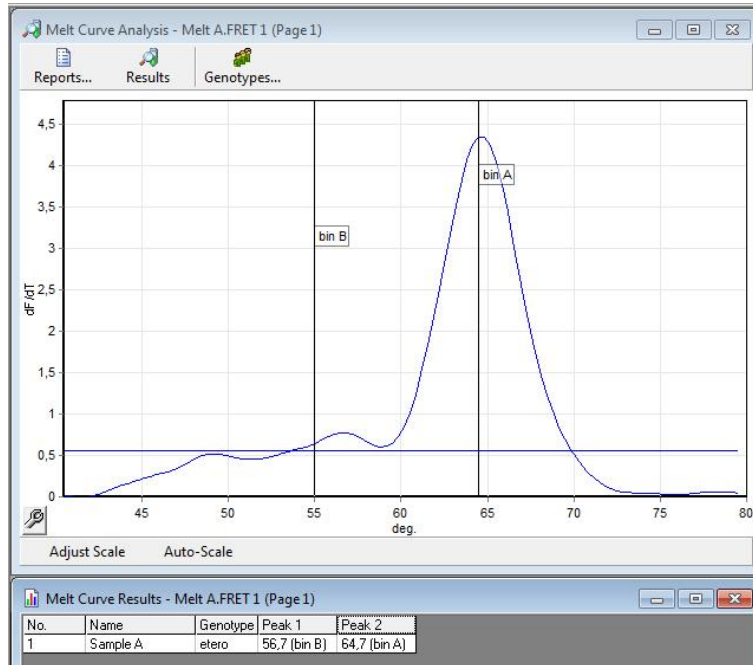
fig. 2 (threshold = 0,5)  $\Rightarrow$  correct typing (homozygous sample)

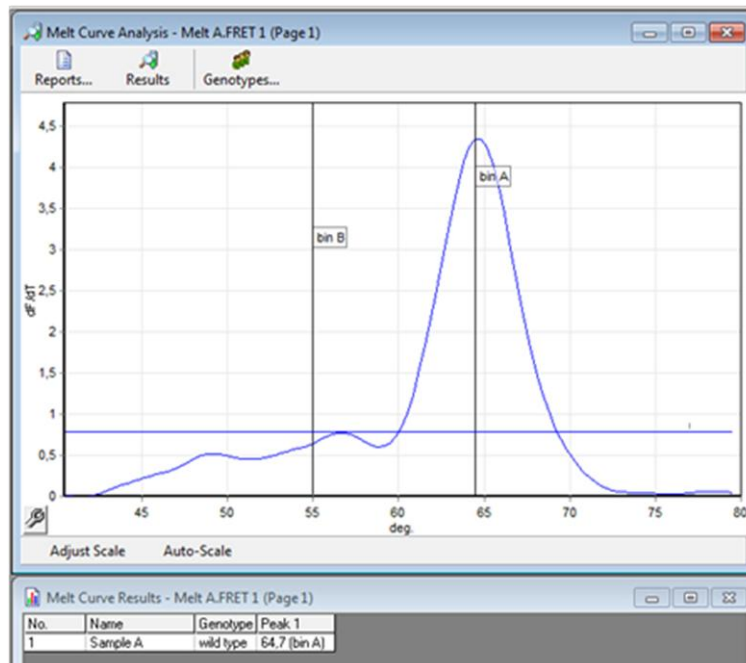- If “peak(s) amplitude” is too low, it’s possible to increase the “gain-value” and repeat the melting step on the same PCR-product:
  - select the “New” icon in order to perform a new assay
  - in the “Profile” function set “thermal-profile” as follow (It takes about 15 minutes):


| Cycle  |
|--|
| Denature: 95°C, 1 min  |
| Hold: 40°C, 1 min  |
| Melt: (40-80°C), hold 15 sec on 1st step, hold 5 sec on next steps, acquiring to Melt A (fret 1) |

- in “View → gain calibration” deselect autocalibration.
- select the “Gain” icon and increase the fret1-value of 1 unit. Start the assay.

### Only for RotorGene 3000

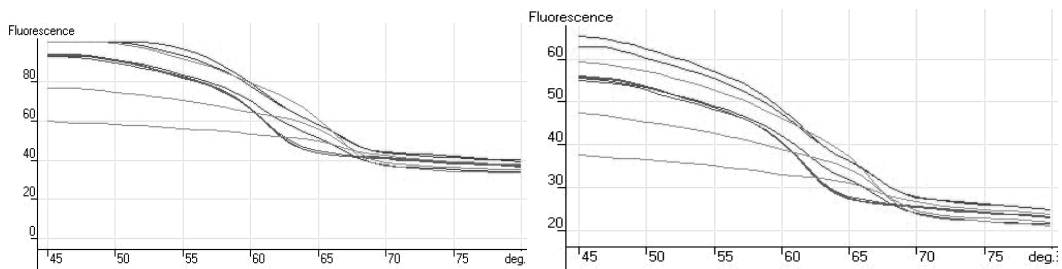If the signals registered during the melting-ramp (raw-data) get to saturation (see fig. on the left) there is a significant risk to lose peaks, resulting in incorrect typing; in this case repeat the assay as previously described decreasing the gain-value ( see fig. on the right).

Example


## CFX

 To use the **CFX96 (Biorad)** instrument, download the user manual cod. NLM **DX007** from the website [www.nlm.it](http://www.nlm.it).

### THERMAL PROFILE

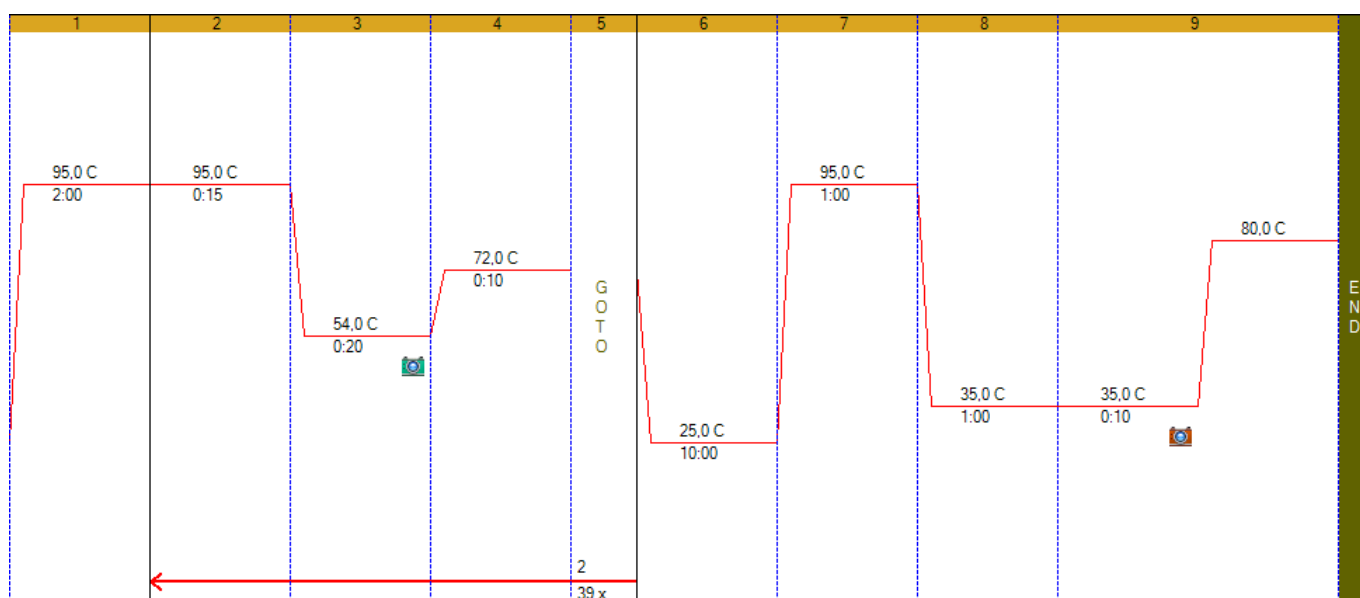- 1 95,0° C for 2:00
- 2 95,0° C for 0:15
- 3 54,0° C for 0:20  
+ Plate Read
- 4 72,0° C for 0:10
- 5 **GOTO 2, 39 more times**
- 6 25,0° C for 10:00
- 7 95,0° C for 1:00
- 8 35,0° C for 1:00
- 9 Melt Curve 35,0 to 80,0° C, increment 0,5° C  
for 0:10 + Plate Read  
END

The image reproduced the temperature profile as shown in the program of the instrument

#### ⇒ Plate setup

- Select *Sample Type* → *Unknown* for samples to be analyzed.
- Select *Target Name* and enter the description shown in the table below for each fluorophore:

| target      | fluorophore |
|-------------|-------------|
| PAI-1 4G/5G | FAM         |

- In case of Positive Controls load, select *Sample Type* → *Positive Control*

## INTERPRETATION OF RESULTS

The window Data Analysis will open automatically at the end of the session.

Move in *Melt Curve* page for the data analysis: a window with the graphic analysis of the assay will be displayed. The graph on the left shows the raw data of the melting-curve, the graph on the right shows

-d(RFU)/dT analysis, with *Melt Peak*.

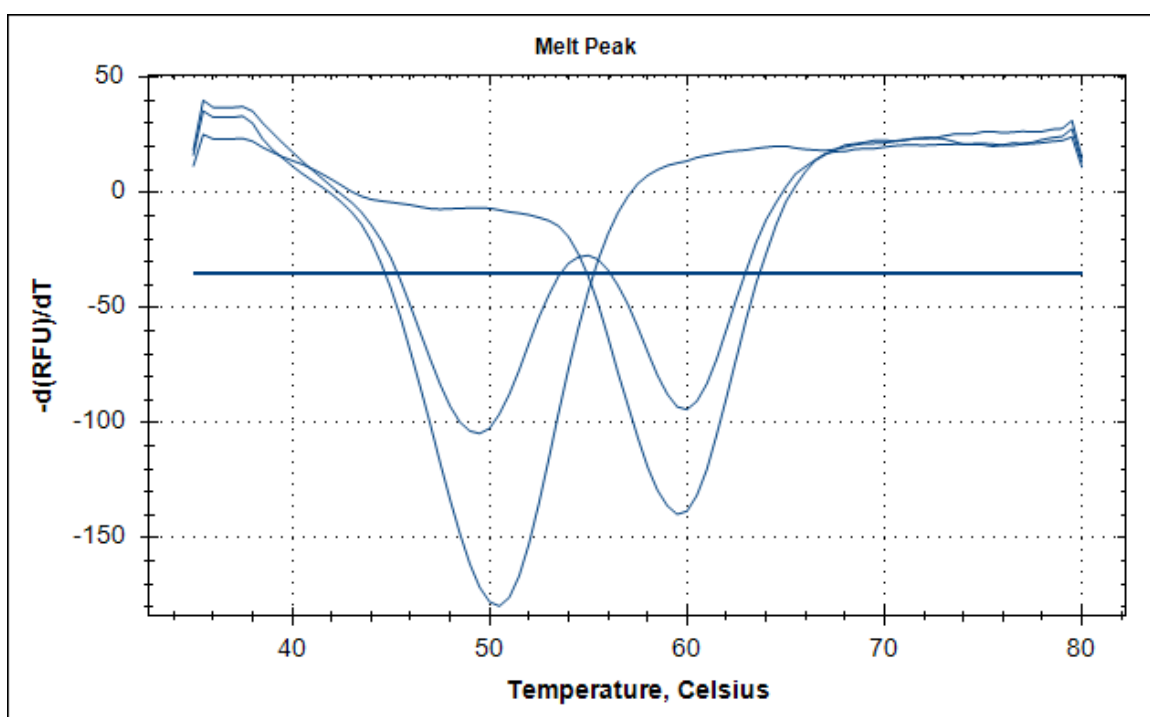
- Select *Peak Type* → *Negative*

The instrument automatically selects the threshold (blue line), but you can move it to remove the background noise and aspecific peaks.

Analyze the *Melt Peak* for each sample by selecting the specific well of the plate.

Each sample could have:

- a single peak at  $54 \pm 2^{\circ}\text{C}$  for 4G homozygous samples
- a single peak at  $61,5 \pm 2^{\circ}\text{C}$  for 5G homozygous samples
- a double peak for heterozygous samples

**Do not consider peaks out of the temperature range.**

Create a Report (*Tools* → *Reports*). Select *Melt Curve* → *Melt Peak Chart* and *Melt Curve* → *Data* in order to obtain the final results. For each sample mark manually the correct genotype.

Use the **Real Gene** program for the automatic results interpretation and the generation of the report, following the manual cod. NLM **DO022**.

- ⇒ The inclusion of the Positive Control in the session is optional. The session is not invalidated by a non-compliant result of the control.

## PERFORMANCE CHARACTERISTICS

### **Analytical Sensitivity**

The analytical sensitivity of the kit "PAI-1 (4G/5G) Real Time", expressed as the minimum quantity of the target that can be detected, is 0,6 ng/µl of DNA (equivalent to 2,4 ng total) and was evaluated by testing serial dilutions of DNA, corresponding to the three different polymorphism, extracted with AA1001 and amplified as method.

### **Diagnostic Sensitivity**

The diagnostic sensitivity of the kit "PAI-1 (4G/5G) Real Time" was assessed by analyzing 60 DNA samples to known genotype (that corresponding to the three different polymorphism, detected by the kit), extracted with AA1001 and AA898. All samples were correctly genotyped, resulting in a diagnostic sensitivity of 100%.

### **Diagnostic Specificity**

The diagnostic specificity of the kit "PAI-1 (4G/5G) Real Time" was determined by analyzing 60 DNA samples to known genotype (that included all three different polymorphism revealed from the kit). Were not obtained false positives. In agreement with these data the specificity of the kit is 100%.

### **Reproducibility**

#### Intra-assay reproducibility

The intra-assay reproducibility was assessed by analyzing 3 samples to different genotype (abstracts with system AA1001 and AA898) in 4 replicates in the same run and on different instruments.

#### Inter-assay reproducibility

The inter-assay reproducibility was assessed by analyzing 3 samples to different genotype (abstracts with systems AA1001 and AA898) in 4 different run, in 4 different days and on different instruments.

On the basis of the results obtained the reproducibility is 100%.

### **WARNING**

As every system based on amplification and detection of nucleic acids, it is possible to have unexpected results in the presence of unknown genetic mutation in the primer and/or probe specific target regions.