BIOLOGIA MOLECOLARE

VER 3 – 13/04/2016

REF AA1302/25

25 TEST

CND W01060103

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CARIO 5 TYPE

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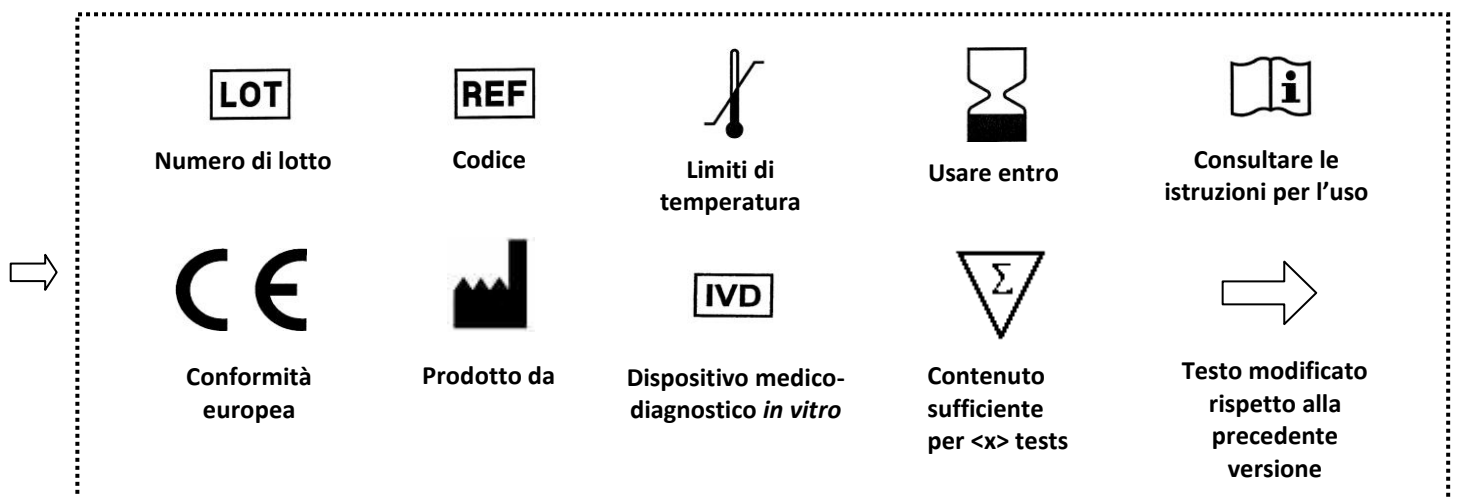
25 TEST

EDMA 16 01 03

● TO BE USED WITH CODE: BA093
EXTRACTION: AA1001/AA340 NOT INCLUDED

CARIO 5 TYPE

**Genetic test for the diagnosis of 13, 18, 21, X and Y
chromosomal aneuploidies**

CE

IVD

NUCLEAR LASER MEDICINE S.r.l.
HEAD OFFICE: Viale delle Industrie, 3 – 20090 SETTALA MI (Italy)
Phone (+39) 02/95. 24. 51 - Fax (+39) 02/95. 24. 52. 37
WEB: www.nlm.it – E-MAIL: segreteria@nlm.it

INTENDED USE

The CARIO 5 TYPE device provides reagents for the characterization of 13, 18, 21, X and Y chromosome aneuploidies through the amplification of 27 chromosome markers (microsatellites or Short Tandem Repeats, STRs) and the subsequent analysis of allelic fragments with an automated Genetic Analyzer.

The assay is compatible with Life technologies Genetic Analyzers (ABI 3500, ABI 3130, ABI 310, ABI 3730) able to work with G5 fluorophores matrix (Dye Set G5) and column-based DNA extraction kits, NLM code AA1001, from amniotic fluid and whole blood (for post-natal diagnosis) and NLM code AA340 (QIAamp DNA Mini kit, Qiagen) from chorionic villus samples.

Amplification enzyme assigned: orange cap.

INTRODUCTION

Chromosome abnormalities represent modifications in number or structure of chromosomes. Numerical chromosome abnormalities, also called aneuploidies, are characterized by a chromosome number alteration compared to standard amount. For example, in case of trisomy, we will find one chromosome more than the standard number ($n=47$). Well-known pathologies like Down's Syndrome (Trisomy 21), Patau's Syndrome (Trisomy 13), Edward's Syndrome (Trisomy 18) belong to this group of diseases.

Regarding X and Y chromosomes, the alteration may be related to the loss or the addition of one of the two chromosomes respect to the normal XX pattern (female) or the normal XY pattern (male). These abnormalities could determine the following diseases: X chromosome monosomy, named Turner Syndrome [45, X_0], Klinefelter Syndrome caused by the presence of a double X chromosome in male subjects [47, XXY], double Y Syndrome [47, XYY] and Triple XXX Syndrome [47, XXX].

Chromosome pattern of polyploidy can also occur and it is characterized by one or more extranumerary haploid chromosome patterns, that are incompatible with life in humans. Triploid syndrome $3n$ [69, XXX] is the simplest form of polyploidy and individuals have three chromosomes for each type.

Trisomies 21, 18, 13 and X, Y aneuploidies represent about 80 - 95% of the possible chromosome aneuploidies and are the most responsible for birth defects. The main mechanism by which aneuploidies are generated is represented by non-disjunction of homologous chromosomes or chromatids during meiosis phase in the gametogenesis. Gametes with an extra and a missing chromosome may form and join to normal gametes, generating respectively a trisomy and a monosomy pattern.

REFERENCES

- *Celia Donaghue et al. J Med Gene. Sep 2004 Supplement, Vol. 41, pS57.*
- *Hulthén, M et al. Reprod. 2003; 126: 279-297.*
- *Nicolini, U et al. Hum Reprod Update. 2004 Vol 10, no.6 pp. 5.*
- *Rahil, H et al. Eur J Hum Genet. 2002 Aug;10(8):462-6.*
- *Schmidt, W et al. Mol Hum Reprod. 2000 Sep;6(9):855-60.*
- *Deutsch, S et al. J Med Genet. 2004; 41: 908-915.*

PRINCIPLE OF THE PROCEDURE

The test is based on multiplex QF-PCR (Quantitative Fluorescence – Polymerase Chain Reaction) that allows the amplification of chromosome markers (Short Tandem repeats, STRs) with fluorescent primers and subsequent fragment analysis on an automated genetic analyzer.

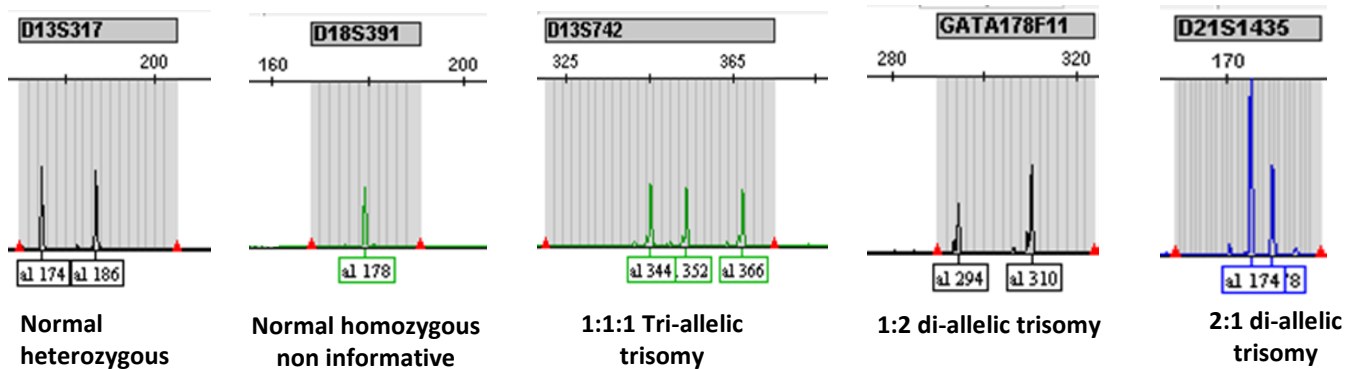
STRs are small nucleotide sequences specific for each chromosome and repeated in variable number in different subjects.

Fragment analysis consists of a capillary electrophoresis of the PCR product through a polymer, resulting in the separation of the different fragments according to their dimension. The fluorescent signal associated to each amplification product is analyzed and displayed as a peak characterized by a specific position and area, which respectively depends on DNA target length and the chromosome number present in the samples.

Normal subjects are often heterozygous for a specific STR and their electrophoretic profile shows two peaks with the same area and height, corresponding to maternal and paternal alleles, respectively.

In monosomy situation or in homozygous subjects, characterized by alleles of the same length, there will be only one peak, which is considered non informative because its quantification is not possible.

In trisomy cases, we can have the presence of three peaks with the same area (in ratio 1:1:1) for tri-allelic trisomy or two peaks, one of which with double area respect to the other, for di-allelic trisomy.

⇒ PRODUCT COMPOSITION

Store at -25/-15°C

Reagents	N° Vials	Volume/Vials
QF-PCR Mix	1	665 µl

STABILITY AND STORAGE

- All the reagents are stable up to the expiry date indicated on the label when stored at the correct temperature (-25/-15°C).
- Thaw Amplification Mix on ice or at +2/+8°C.
- Fluorescent primers are photosensitive: avoid direct and prolonged exposure of the **Mix** to light.
- QF-PCR **Mix** has to be used immediately after preparation; once dispensed in PCR tubes, the remaining reagents have to be discarded. Avoid prolonged exposure to light.
- Keep amplification reagents on ice during the whole procedure.
- At the end of each assay store reagents back at their correct storage temperature.

PRECAUTIONS

- Only professional and opportunely trained personnel should use this kit. Handle this product according to established Good Laboratory Practices and universal precautions; wear personal protective apparel.
- Use aerosol-resistant pipette tips to avoid pipettes contamination.
- Discard used materials as bio-hazardous waste.
- Do not eat, drink, smoke or apply cosmetics in areas where reagents or specimens are handled.
- If skin or mucous membrane exposure occurs, immediately wash the area with copious amount of water. Seek medical advice.
- Do not use components beyond the expiration date.
- Do not mix reagents from different lots.
- Reagents have to be preserved separated from possible contaminants (as DNA samples and amplification products).
- Avoid repeated freezing and thawing of the mix.
- It is recommended to make analysis in three different areas:
 - Area 1: pre-PCR (samples handling and extraction)
 - Area 2: Mix preparation
 - Area 3: post-PCR (Amplification and fragment analysis)
- Don't use the device if the box is damaged; contact the supplier.
- **It is advisable to have constant and uniform laboratory temperature, avoid to place the instruments near heating/cooling sources that may compromise the correct work.**

MATERIALS REQUIRED BUT NOT SUPPLIED

ZONE 1

DNA Extraction Kit
Vertical downflow airbox
Dedicated adjustable volume pipettes set
Aerosol-resistant pipettes tips
0,2 ml PCR tubes
Spectrophotometer

ZONE 2

Vertical downflow airbox
Dedicated adjustable volume pipettes set
Aerosol-resistant pipettes tips
DNA polymerase with orange cap

⇒ ZONE 3

Thermalcycler (AB2720, AB9700, Veriti, SimpliAmp - Applied Biosystems; Mastercycler - Eppendorf; MyCycler, T100™, C1000, Biometra T Personal - BioRad; or similar thermalcycler)*
Genetic Analyzer, Life Technologies (ABI 3500/3130/310/3730)
96 well plate (0,2 ml) for genetic analyzer
Polymer: POP7 (ABI 3500/3130/3730), POP4 (ABI 310)
GeneScan 600 LIZ Size Standard V2.0
Hi-Di Formamide
DS-33 (Dye set G5) Matrix Standard kit (for genetic analyzer calibration)
DS-33 GeneScan Installation Standard (only for ABI 3500)

* **WARNING:** if thermal cyclers different from those mentioned above are used, verify that the ramp rate (heating/cooling) is $\leq 3^{\circ}\text{C}/\text{sec}$. For T100™ (BioRad) set ramp rate as default.

PROCEDURE

DNA EXTRACTION

DNA Isolation kit compatible:

- NLM code **AA1001** for DNA extraction from amniotic fluid (AF) and whole blood (for post-natal diagnosis).
Start from an AF volume of 2 ml, centrifuge at maximum speed for 10 minutes, discard the supernatant, leaving 200 µl of liquid to resuspend the pellet. Follow the manufacturer instructions for use and at the end, if the starting pellet was clearly visible, resuspend the DNA in 100 µl of elution buffer, otherwise resuspend it in 50 µl.
- NLM code **AA340** (QIAamp DNA Mini kit, Qiagen) for DNA extraction from chorionic villus or abortion material biopsy.
In a 1,5 ml tube containing 10-25 mg of chorionic villus or abortion material biopsy add 80 µl of sterile PBS. Mix well with a tip to homogenize the sample and partially degrade the biopsy. Add 100 µl of lysis buffer (ATL) and proceed as indicated in the instructions for use. According to the starting amount of biopsy resuspend the pellet in a final volume of 100 or 200 µl of elution buffer (AE).

Warning

- It is possible to use biological samples stored at -25/-15°C.
- Avoid using biological samples or DNA after repeated freeze/thaw cycles or not correctly stored in order to prevent bad assay results.
- For whole blood use EDTA or citrate only as anticoagulant, do not use heparin.

AMPLIFICATION

Prepare the QF-PCR Mix as follows:

Reagents	Volume/sample
Mix	21,75 µl
DNA polymerase (5U/ µl), orange cap	0,25 µl

- Prepare the Mix for the number of samples to test + 1 more volume (if "n" ≤ 10) or + 2 volumes (if "n" >10).
- Mix gently and dispense **22 µl** of the Mix in the previously marked 0,2 ml test tubes.
- Add **1-3 µl** of DNA from chorionic villus or abortion material biopsy or **2-3 µl** of DNA from amniotic fluid samples to each tube (depending on the DNA yield of purification method used) and mix pipetting up and down.
- The recommended range of the DNA concentration is between 3-30 ng/µl

⇒ **Alternatively, it is possible to add the DNA Polymerase directly to the amplification Mix** and aliquot it in 25 tubes of 0,2 ml:

- After thawing the Mix, centrifuge it briefly before opening, to break down any droplets present in the cap;
- Add **7,7 µl** of DNA Polymerase;
- Mix gently pipetting several times and aliquot **22 µl** in the 25 tubes of 0,2 ml;
- Store the tubes at **-25/-15°C** for at least 60 days. During this time take them when you need them, thaw them, mark them with the name of sample, add the volume of DNA required and proceed with the amplification.

Thermal Profile Setting

Put the reaction tubes in the thermal cycler and run the following amplification program:

Temperature	Time	Cycles
95°C	1 min	1
95°C	40 sec	27
60°C	90 sec	
72°C	40 sec	
60°C	45 min	1
10°C	∞	-

Set the reaction volume to **25 µl** and start the amplification.

DETECTION

Genetic Analyzer setting

- To use the Genetic Analyzer follow the manufacturer instructions for maintenance and equipment management.
- Carry out the spectral calibration with DS-33 matrix (Dye Set G5).

Run setting

Warning: Before starting the run, set the oven temperature to 60°C.

- ✓ For **ABI 3500** genetic analyzer set the sample sheet from the Dashboard as follows:
 - Insert the sample name
 - Associate the specific "Assay": Run Module ABI 3500:
 - Instrument Protocol: Fragment Analysis 50_POP7_1
 - Size Calling Protocol: Fragment_Analysis_PA_Protocol

Instrument Protocol	ABI 3500
Capillary length	50 cm
Polymer	POP-7
Dye Set	G5
Run Temperature	60°C
Run Time	1330 sec
Injection Time	10-15 sec*
Run Voltage	19,5 KVolts

*The Injection Time is indicative. It might be necessary to modify it due to the quantity and quality of the DNA, which depend on the type of DNA extraction used.

- Associate the "File Name Convention"
- Associate the "Result Group"

✓ For **ABI 3130** genetic analyzer specify:

- NEW MODULE: indicate the run parameters;
- NEW PROTOCOL: define the “Run Module” and the “Dye Set G5”;

Apply to Run Module the following parameters:

Instrument Protocol	ABI 3130
Capillary length	36 cm
Polymer	POP-7
Dye Set	G5
Run Temperature	60°C
Run Time	1300-1800 sec
Injection Time	35 sec*
Run Voltage	15 KVolts

* The Injection Time is indicative. It might be necessary to modify it due to the quantity and quality of the DNA, which depend on the type of DNA extraction used.

- NEW PLATE.

✓ For **ABI 310** genetic analyzer specify the following parameters:

Instrument Protocol	ABI 310
Polymer	POP-4
Dye Set	G5
Run Temperature	60°C
Run Time	35 min
Injection Time	5 sec*
Run Voltage	15 KVolts

* The Injection Time is indicative. It might be necessary to modify it due to the quantity and quality of the DNA, which depend on the type of DNA extraction used.

⇒ ✓ For **ABI 3730** genetic analyzer specify the following parameters

- NEW MODULE: indicate the run parameters;
- NEW PROTOCOL: define the “Run Module” and the “Dye Set G5”;
- Apply to Run Module the following parameters:

Instrument Protocol	ABI 3730
Capillary length	36 cm
Polymer	POP-7
Dye Set	G5
Run Temperature	60°C
Run Time	1800 sec
Injection Time	6-15 sec*
Run Voltage	15 KVolts

(*) *The Injection Time is indicative. It might be necessary to modify it due to the concentration and quality of DNA, which depend on the type of extraction method used.*

- NEW PLATE.

Sample preparation

- Prepare a solution with **12 µl** Formamide and **0,3 µl** Gene Scan 600 LIZ v2.0 for each sample and pipette **12 µl** of this solution in each well of the plate.
- Centrifuge briefly the PCR products and open the tubes carefully to avoid possible contaminations.
- Add **1 µl** (or **2 µl** on **ABI 3130 Genetic Analyzer**) of amplification product to each well.
- Centrifuge briefly the plate to have all the solution on the bottom of each well without any bubbles, later close it with the specific septa 96 well.
- Put the plate at 95°C for 2 minutes for amplicons denaturation and then cool it on ice for 5 minutes.
- Close the plate in the specific 96 well retainer with base set and put it in the Genetic Analyzer (when only one plate is uploaded, use always the position “A” of the tray).
- Click on “Start Run”.

It is possible to monitor the run and evaluate the raw data in the Data Collection. For the analysis use the GeneMapper Software v4.1 or v4.0.

INTERPRETATION OF RESULTS

CARIO 5 TYPE device allows the simultaneous analysis of 27 chromosome markers: 6 specific markers for chromosome 21, 6 for chromosome 13, 6 for chromosome 18, 9 for chromosomes X and Y, of which 2 (SRY, DYS448) are specific for Y chromosome only (therefore absent in female samples).

Each marker is identified according to the dimension (bp) and the color of the respective amplicon (amplicons with similar length are distinguished by different fluorophores). In the following table the parameters of each marker are listed:

MARKER	CHROMOSOME	Dye	MARKERS RANGE (bp)
AMG	X,Y	blu	116 - 130
TAF9L	3,X	blu	135 - 145
D21S1435	21	blu	155 - 188
X22	X,Y	blu	190 - 250
D18S390	18	blu	262 - 310
D21S147	21	blu	316 - 348
SBMAbis	X	blu	351 - 440
SRY bis	Y	blu	460 - 480
D18S391	18	green	163 - 191
DXS8377	X	green	200 - 255
D21S1411	21	green	262 - 314
D13S742	13	green	315 - 377
D13S305	13	green	405 - 455
D21S1442	21	green	481 - 600
D18S535	18	yellow	123 - 164
7X_3	7,X	yellow	170 - 210
D13S258	13	yellow	222 - 286.8
GATA178F11	18	yellow	288 - 323.8
D18S386	18	yellow	324 - 395
D13S317	13	yellow	400 - 438
D13S634	13	yellow	440 - 510
D21S11	21	red	202 - 255
HPRT	X	red	258 - 278
D18S1002	18	red	280 - 315
D21S1437	21	red	316 - 376
DYS448	Y	red	377 - 415
D13S628 bis	13	red	420 - 480.8

Table 1. Marker list. Markers range (bp) may vary according to the type of polymer used for electrophoresis.

⇒ To perform the “CARIO 5 TYPE” analysis import in GeneMapper software (v4.1 or v4.0) the following **NLM** parameters, provided with the CD:

- Analysis Method
- Panel
- Size Standard
- Matrix File “.mtx” extension (ABI 310 only)

To analyze the single run proceed as follows:

- Import Run Files with the “.fsa” extension;
- For each sample apply the parameters previously indicated;
- Save and start the analysis;
- Select the sample in order to view the electrophoretic profile. Cut the first region of the electropherogram (which corresponds to the background noise due to the considerable amount of fluorophores in the mix). Set the display with separate fluorophores and reduce the Y scale axis to increase the amplitude of the peaks.
- For each fluorophore move the peaks threshold line to eliminate the possible residual background.
- The final electrophoretic profile gives information about all the markers for the sample analyzed.
ABI 3500/3730: the acceptable range of the peaks is 175 - 32000 rfu. Over 32000 rfu the saturation of the CCD camera occurs and the signal is considered out of scale. The best range is between 175 and 12000 rfu.
ABI 3130: the acceptable range of the peaks is between 50 and 8000 rfu.
- For each chromosome evaluate the state of zygosity through the analysis of the related markers. The result is considered normal when at least two markers are informative, that means a di-allelic genotype (1:1).

For each sample start the analysis with the evaluation of Amelogenin marker (**AMG**, a gene that allows sex discrimination on the basis of the number of peaks), TAF 9L (**T.**, a marker with paralogous sequences on chromosomes 3 and X) and 7X (a marker with paralogous sequences on chromosomes 7 and X) to discriminate female XX samples from females with X monosomy (Turner Syndrome [45, X₀]).

- **Figure 1, page 10:** In a normal female subject (**46 XX**), TAF9L and 7X markers are expected to show a 1:1 ratio between the peaks areas, only one peak for AMG and no peaks for SRY and DYS448 markers (specific for males).
- **Figure 2, page 10:** In a normal male subject (**46 XY**), TAF9L and 7X markers are expected to have a 2:1 ratio between the peaks areas, two peaks with the same height and area for the AMG marker and only one peak for SRY and DYS448 markers.
- **Figure 3, page 11:** In a female with X monosomy (**45, X₀**), the TAF9L and 7X markers are expected to show a 2:1 ratio between the peaks areas, only one peak for AMG marker and no peaks for SRY and DYS448 markers.

Statistical analysis

According to the QF-PCR guidelines, the analysis of heterozygous di-allelic markers is based on calculation of the ratio between the peaks areas of the two alleles (A1/A2):

	1:1 (normal heterozygous)	2:1 or 1:2 (di-allelic trisomy)	1:1:1 (tri-allelic trisomy)**
A1/A2* (distance between the peaks < 24 bp)	0,8-1,4	1,8-2,4 (A1>A2) ; 0,45-0,65 (A1<A2)	0,8-1,4
A1/A2* (distance between the peaks ≥ 24 bp)	0,8-1,5		0,8-1,5

*A1 and A2 represent the areas of the two alleles, respectively.

** in case of trisomy 1:1:1 consider also the A1/A3 and A2/A3 ratios.

If this areas range is not respected, it is possible to calculate the ratio between alleles height, considering the same criteria previously indicated.

For normal di-allelic heterozygous (1:1) markers, if the ratio A1/A2 falls out of the range of 0.8-1.4, it may be due to several factors, including:

- whole chromosome trisomy;
- partial chromosome trisomy (including sub-microscopic duplications);
- mosaicism;
- second contaminating genotype (e.g. maternal, twin, external);
- “stutter” (peaks smaller than the main one) causing skewing (significant deviations from 1:1 ratio);
- preferential amplification of one allele causing skewing;
- primer site polymorphisms;
- somatic microsatellite mutations.

To perform the statistical analysis of heterozygous di-allelic markers, it's possible to export the file.txt, generated from GeneMapper, and convert it in Excel format. Then, setting the calculation formula, it is possible to obtain automatically the ratio between the peaks areas of the two alleles A1/A2.

Some electropherograms are suggested as examples:

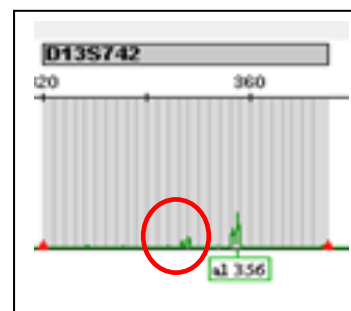
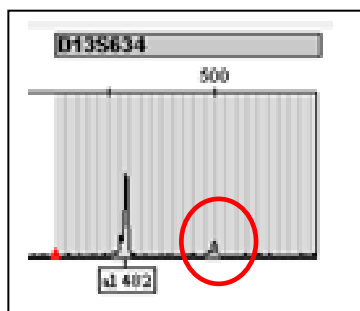
- **Figure 4, page 12: Male [46 XY]**
- **Figure 5, page 13: Female [46 XX]**
- **Figure 6, page 13: Trisomy 21 [47 XY,+21]**
- **Figure 7, page 14: Trisomy 18 [47 XX,+18]**
- **Figure 8, page 14: Trisomy 13 [47 XY,+13]**
- **Figure 9, page 15: Klinefelter syndrome [XXY]**
- **Figure 10, page 15: Triploidy syndrome [69,XXX]**
- **Figure 11, page 16: Turner syndrome [45, X₀]**
- **Figure 12, page 16: Double Y syndrome [XYY]**
- **Figure 13, page 17: Triple X syndrome [47,XXX]**

TROUBLESHOOTING

If the starting DNA is too concentrated (DNA concentration >30 ng/μl), it is possible to observe a saturation of the signal of the electrophoretic profile. In this case, it is recommended to dilute the DNA before the amplification or the PCR product, and then repeat the run on the genetic analyzer.



PCR Artefacts

Aspecific peaks, named “**Stutter peaks**”, are displayed as “extra peaks” (skewing). It is a repetition smaller than the real allele caused by an error of Taq Polymerase during PCR amplification of repeated sequences. Typically the area of these aspecific peaks is 15% of the area of corresponding STRs. This artefact is rare and imperceptible in case of penta and tetra-nucleotides repetitions, but it may increase in case of tri and di-nucleotides repetitions.

Artefacts during the detection

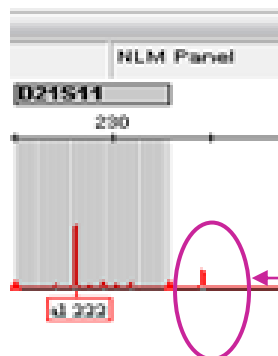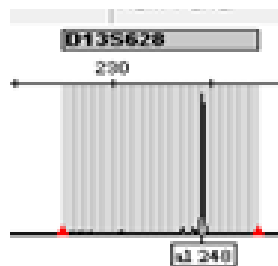
- Sometimes, when a marker shows a very intense signal, crosstalk between dye-channels may occur during the detection (“peak under peak”). Crosstalk peaks must be excluded from the analysis.

Crosstalk peak between yellow channel (black peak) and red channel. This “peak under peak” is perfectly overlapped to the black one and it must be excluded from the analysis.

Critical point for D13S634 marker:

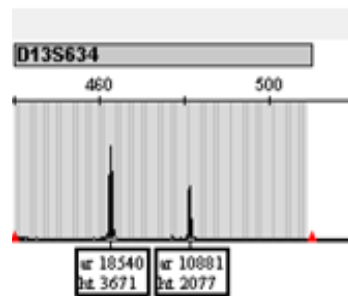It is possible that this chromosomal marker gives an imbalanced signal of the peaks in di-allelic heterozygous samples. In this case, it is necessary to analyze all the markers for the chromosome 13 in order to diagnose the sample correctly. In relation to the guidelines, only the presence of at least two abnormal markers can identify a chromosomal alteration.

**PERFORMANCE CHARACTERISTICS****Diagnostic specificity**

The specificity of the “CARIO 5 TYPE” assay was determined analyzing 145 normal samples: fetal DNA extracted from Amniotic Fluid (AF), Chorionic Villus Samples (CVS), abortion material and whole blood. No false positive results were observed, therefore “CARIO 5 TYPE” Test Specificity is 100%.

Analytical sensitivity

The analytical sensitivity of the “CARIO 5 TYPE” assay, expressed as the minimum amount of target DNA that can be exactly detected, is 3 ng/μl.

Diagnostic sensitivity

The diagnostic sensitivity of the “CARIO 5 TYPE” assay was evaluated analyzing 63 DNA samples with known 13, 18, 21 X and Y chromosome alteration (see the following table). The device discriminated exactly all the pathological samples, therefore the diagnostic sensitivity is 100%.

Pathological samples used:

Nr. of samples tested	Chromosome Alteration
29	Trisomy 21
15	Trisomy 18
7	Trisomy 13
5	45, X ₀ (Turner)
2	XXY (Klinefelter)
1	XYY (Double Y)
2	47 XXX (Triple X)
2	69 XXX (Triploidy)
63	Total

Reproducibility

INTRA-ASSAY REPRODUCIBILITY (within series precision):

Intra-assay reproducibility was evaluated using 3 samples with different chromosome alterations in 2 replicated. The same 3 samples were tested with two different batches of “CARIO 5 TYPE” assay.

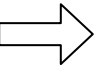
INTER-ASSAY REPRODUCIBILITY (series to series precision):

Inter-assay reproducibility was evaluated using 3 samples with different chromosome alterations by two different operators in different runs, using two different batches.

The test was performed according to the instructions for use and the reproducibility is 100%.

VIC, NED and PET fluorophores, included in the mix, are trademarks of Life Technologies Corporation and their use is protected by patents of Applied Biosystems, LCC.

 Lot Number	 Catalogue number	 Temperature Limitation	 Use by	 Consult Instructions for use
 European Conformity	 Manufacturer	 <i>in vitro</i> diagnostic service	 Sufficient for	 Modified text compared with the previous version