Istruzioni d'uso

ver. 9 - 12/05/2022

Biologia Molecolare

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CVD6 Multiplex Real Time

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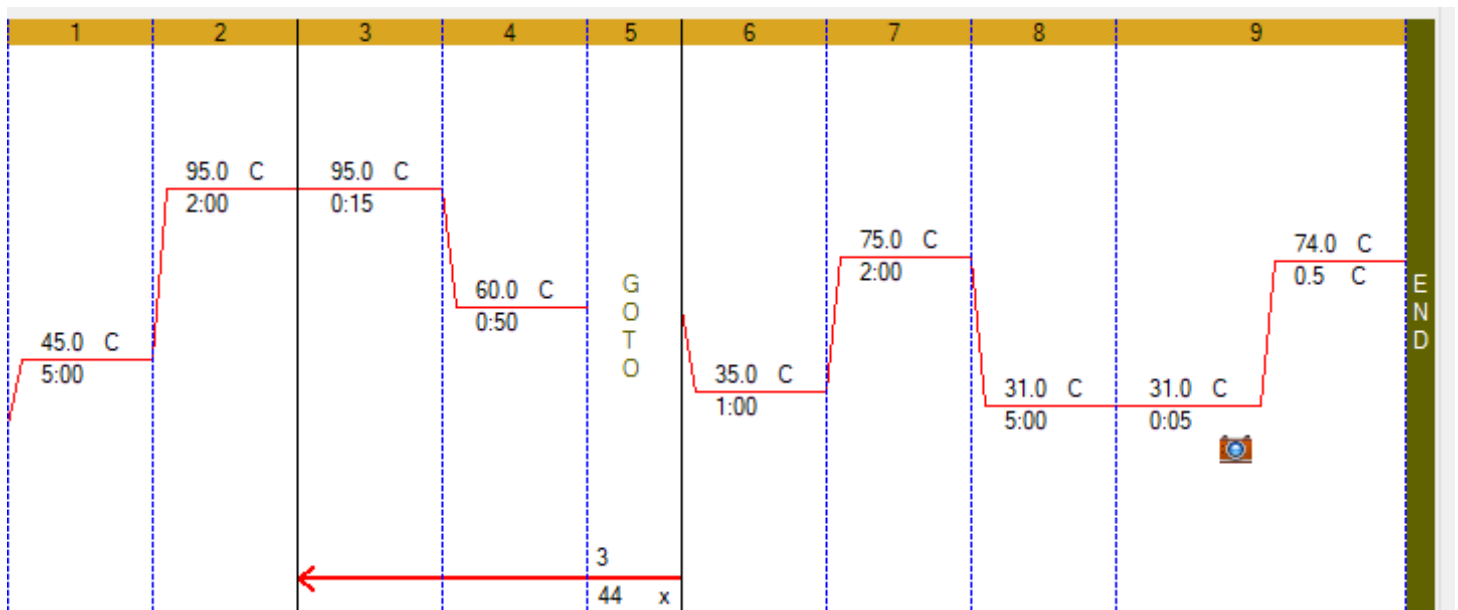
```
1 45,0° C for 5:00
2 95,0° C for 2:00
3 95,0° C for 0:15
4 60,0° C for 0:50
5 GOTO 3, 44 more times
6 35,0° C for 1:00
7 75,0° C for 2:00
  Slow Ramp Rate to 0,1°C per second
8 31,0° C for 5:00
9 Melt Curve 31,0 to 74,0° C, increment 0,5° C
  for 0:05 + Plate Read
  END
```

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The screenshot displays the 'Plate Loading Guide' software interface. The main area is a 96-well plate grid with columns numbered 1 to 12 and rows labeled A to G. The C row is highlighted in blue. Wells C3 through C10 contain target names and fluorophore assignments. The right-hand panel shows configuration options: Sample Type (Unknown), Target Names (FAM, HEX, ROX, Cy5, Cy5.5), Sample Names (7 petr), Biological Group (<none>), and Replicate # (1).

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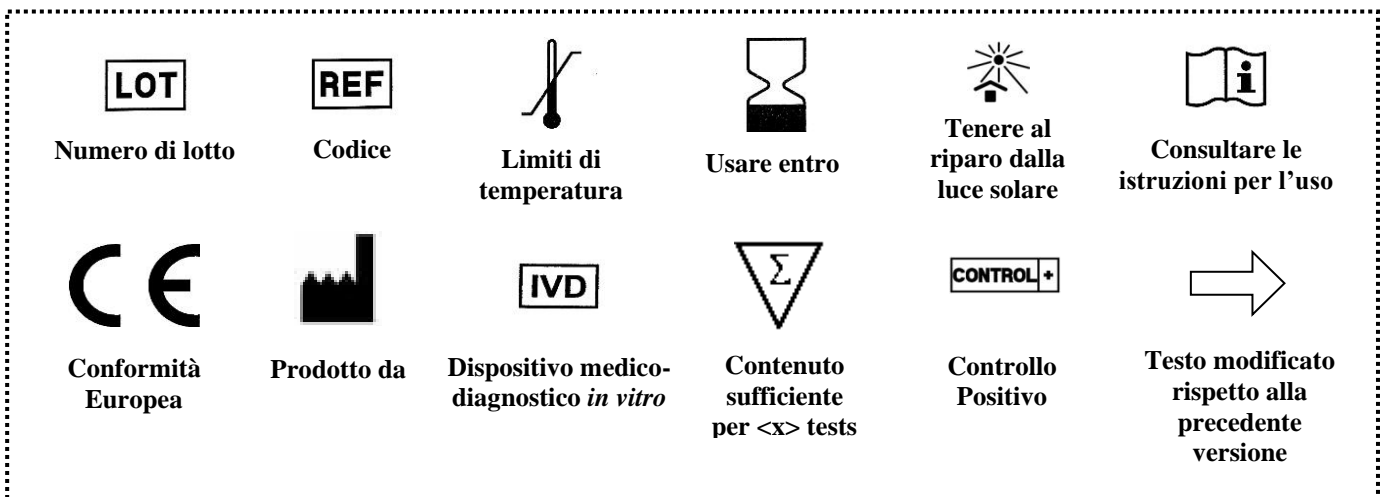
Extraction not included

CVD6 Multiplex Real Time

Analysis of Factor V G1691A, Factor V H1299R, Factor II G20210A,
MTHFR C677T, MTHFR A1298C, PAI-1 4G/5G variants

REF

AA1397/48

CE

48 TEST

GMDN

59910

IVD

NUCLEAR LASER MEDICINE S.r.l.

HEADQUARTER: Via Cascina Conighetto – BUSINESS OFFICES: Viale delle Industrie, 3 - 20049 SETTALA (MI), ITALY

Phone: (+39) 02/95.24.51 - Fax (+39) 02/95.24.52.37 - (+39) 02/95.24.52.38 WEB: www.nlm.it - E-MAIL: segreteria@nlm.it

ISO 9001 Quality Management System Organization certified and ISO 13485 Medical Sector Quality Management System certified (IMQ certification body - CSQ certification).

INTENDED USE

The **AA1397/48** kit allows the simultaneous analysis of six different mutations and polymorphisms involved in deep venous and arterial thrombosis: Factor V G1691A, Factor V H1299R, Factor II G20210A, MTHFR C677T, MTHFR A1298C, PAI-1 4G /5G. Genotyping is performed by amplification of target sequences and melting curves analysis.

This device has been validated with:

a) the on column manual extraction kit (AA1001 NLM code), the OMNIA LH75 automatic extraction kit (AA1319/48 NLM code) and with the most common automatic nucleic acid extraction systems (AA1185 NLM code Magcore extraction kit; AA1038 NLM code Maxwell extraction kit; AA1439/192 NLM code Qiasymphony extraction kit).

b) the CFX Real Time PCR (BioRad) instrument.

Amplification enzyme and Uracil-DNA glycosylase (UNG) to prevent contamination from amplified are included in the mix.

INTRODUCTION

Venous thrombosis is among the three most common cardiovascular diseases and involves approximately 1 in 1000 individuals each year. Thrombotic risk is due to both environmental factors (age, surgery, pregnancy, oral contraception) and genetic predisposition. For example, the G>A mutation at position 1691 in the coagulation Factor V gene (FV Leiden - R506Q) was found with a high frequency (20-60%) in patients with thrombosis. Other mutations in the Factor V gene, such as the HR2 haplotype (H1299R) which involves the replacement of A>G at nucleotide 4070, results in thrombosis especially if associated with the R506Q mutation. Point mutations in methylenetetrahydrofolate reductase (MTHFR C677T and A1298C) and in the 3'-UTR region of the prothrombin gene (Factor II G20210A) have been associated respectively to a reduction in the enzymatic activity of MTHFR and to a high prothrombin level in plasma. Each of these variants, in association with the Leiden Factor V variant determines an increased risk related to venous thromboembolism.

Among potential genetic markers of cardiovascular risk was also identified the PAI-1 (Plasminogen Activation Inhibitor - type 1) gene that plays a critical role in the regulation of intravascular fibrinolysis. The presence of a 4G sequence (instead of 5G) within the promoter region was associated with an increased risk of venous thromboembolism.

REFERENCES

- Faioni *et al*, Blood 1999; 94 (9): 3062-6
- Doggen *et al*, Circulation 1998; (97): 1037-1041
- De Stefano *et al*, Blood 1998; 91 (10): 3562-3565
- Wald *et al*, BMJ 2002; (325): 1202-1208
- Gemmati *et al*, Haematologica 1999; (84): 824-828
- Gemmati *et al*, Wound Rep Reg 2004; 12 (5): 512-517
- Per-Gunna *et al*, Stroke 2005; (36): 1661-1665

PRINCIPLE OF THE PROCEDURE

The test is based on the Real Time PCR technique in which the extracted DNA is amplified in the presence of a probe complementary to the region that includes the mutation: the probe is labeled in 5' with a specific fluorophore while in 3' a quencher is bound. When the probe is not bound to the DNA the fluorescence emitted by the fluorophore is dampened by the quencher: in the absence of the target there is therefore no fluorescence emission. However, when the probe appears in the DNA, the fluorophore is no longer damped by the quencher and, in the presence of a light source, fluorescence is emitted at a specific wavelength.

The probes allow to identify variants of the target sequence by means of the dissociation curve (melting curve), subjecting the amplified to a rising temperature ramp with consequent detachment of the probes from the DNA. If the probe is complementary to the mutated DNA strand, the hybridization between the two will be more stable than the duplex between the probe itself and the strand that does not carry the mutation (single point mismatch), resulting in a peak of dissociation at a higher temperature. If, on the other hand, the probe is complementary to the strand that does not carry the mutation, the wild type peak will have the peak of dissociation at the highest temperature.

By melting curve analysis, the three genotypes can therefore be distinguished based on the position of the peaks: single peaks at two different temperatures for the homozygous (normal/mutated) samples and double peak for the heterozygous ones.

The analysis includes two successive phases:

1. Genomic DNA extraction.
2. Amplification and detection of target sequences using Real Time PCR.

The regions of the MTHFR, FV, FII and PAI-1 genes that comprise the respective mutations of interest are co-amplified in a single Real Time PCR reaction using dedicated selected primers.

⇒ The individual variants are analyzed through the use of specific probes linked to different fluorophores:

- MTHFR C677T: the specific probe is linked to the FAM fluorophore
- MTHFR A1298C: the specific probe is linked to the ROX fluorophore
- PAI-1 4G / 5G: the specific probe is linked to the CY5 fluorophore
- FV H1299R: the specific probe is linked to the CY5.5 fluorophore
- FV G1691A: the specific probe is linked to the HEX fluorophore
- FII G20210A: the specific probe is linked to the ROX fluorophore

The CFX (BioRad) includes a thermocycler and a fluorescence detection device in a single instrument. The computer connected to the system collects and displays the fluorescence data in the graph shown in the manager instruments software. The data analysis is performed by the Real Gene software (DO022 NLM code).

⇒ PRODUCT COMPOSITION (Store at -25/-15°C)

Reagents	NLM Code	N° vial	Volume/vial	Warning
MasterMix CVD6 , blue cap Solution containing oligonucleotides, fluorescent probes and enzymes	KA1329-n	2	300 µl	
Water DNase, RNase, Protease free	AZ040-n	1	500 µl	-

The amount of reagents indicated in the table is referred to the standard size of the kit.

Reduced packaging is available on request for evaluation and/or demonstration of the product.

STABILITY AND STORAGE

- All the reagents are stable up to the expiry date indicated on the label when stored at the correct temperature (-25/-15°C).
- Thaw Amplification Mix on ice or at +2/+8°C
- Probes contained in the **MasterMix CVD6** tube are photosensitive: avoid direct and prolonged exposure to light.
- Keep amplification reagents in ice during the entire procedure.
- At the end of each assay store reagents back at their correct storage temperature.

PRECAUTIONS

- The procedure should be performed by qualified personnel according to Good Laboratory Practices and using usual personal protective equipment.
- Use filter tips to avoid pipettes contamination.
- Discard the disposable material used, gloves and all reagents used as bio hazardous waste.
- Do not eat, drink, smoke or apply cosmetics in areas where reagents or specimens are handled.
- If skin or mucous membrane exposure occurs, immediately wash the area with copious amount of water. Seek medical advice.
- Do not use components after expiration date.
- Do not mix reagents from different lots.
- Reagents have to be preserved from possible contaminants (as DNA samples and amplification products)
- Avoid repeated freezing and thawing of the mix.
- It is recommended to make analysis in three different areas:
 - Area 1: pre-PCR (samples handling and extraction)
 - Area 2: Master Mix preparation.
 - Area 3: post-PCR (Real Time PCR)
- Do not use the device if box is damaged; contact the supplier.
- It is advisable to have constant and uniform laboratory temperature, avoid to place the instruments near heating/cooling sources that may compromise the correct work.

MATERIALS REQUIRED BUT NOT SUPPLIED

ZONE 1

DNA Extraction Kit
Vertical downflow airbox
Dedicated adjustable volume pipettes set
Filtered pipette tips and disposable tubes

ZONE 2

Vertical downflow airbox
Dedicated adjustable volume pipettes set
Filtered pipette tips
0,2 ml Real Time PCR strips or plates

 Heterozygous positive control, single use (cod. NLM FA103/10).

ZONE 3

Real Time Thermalcycler Instrument: 5 channel CFX BioRad instrument with manager software.

 Real Gene interpretive software (DO022 NLM code).

⇒ For compatibility with other instruments, contact *Nuclear Laser Medicine srl*.

PROCEDURE

DNA EXTRACTION

Compatible extractions:

- manual column-based extraction (AA1001 NLM code)
- automatic extraction with OMNIA LH75 System (AA1319/48 NLM code) or with the most common automatic nucleic acid extraction systems (AA1185 NLM code Magcore extraction kit; AA1038 NLM code Maxwell extraction kit; AA1439/192 NLM code Qiasymphony extraction kit – Protocol: DNA Blood 200; Elution volume: 200 μ l).

For the use of the automatic system, follow the instructions provided by Nuclear Laser Medicine srl.

Use fresh or correctly stored (+2/+8°C for up to 2 days or -25/-15°C for longer periods) blood samples. Use only EDTA or Citrate as anticoagulants, not heparin.

AMPLIFICATION

Warning:

- Avoid using repeated freeze/thaw cycles or not correctly stored blood and/or isolated DNA samples in order to prevent bad assay results.
- Use dedicated Real Time PCR disposable tubes only
- Gently shake the Mastermix before use and check that there are no residues in the cap
- DNA concentration to be used in this test should be 15-60 ng/ μ l)
- The Positive Control is for single use only. Discard the remaining material in the tube.

⇒ Manual PCR master mix setup

Reagents	Volume/sample
MasterMix CVD6	10 μ l
Water DNase, RNase, Protease-free	6 μ l
DNA	4 μ l

Automatic PCR master mix setup

Use the specific protocol for the automatic instrument used, provided by *Nuclear Laser Medicine srl*.

THERMAL PROFILE SETUP AND RESULTS ANALYSIS

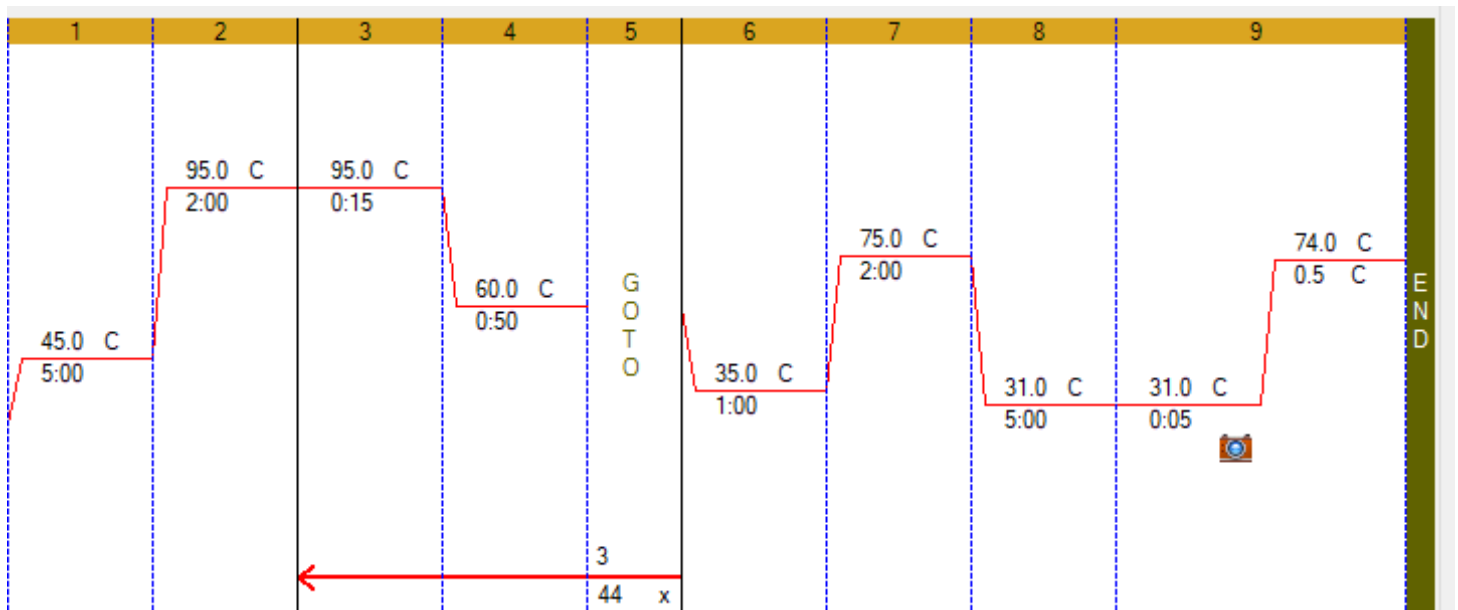
 For **CFX96 (Biorad)** instrument use, download the Instructions for Use (NLM code **DX007**) from the www.nlm.it. For compatibility with other Real Time PCR instruments, contact *Nuclear Laser Medicine srl*.

⇒ RUN SET UP

- Enter the reaction volume (**20 µl**) in the “*Sample Volume*” box
- Enter the following thermal profile:

```

1  45,0° C for 5:00
2  95,0° C for 2:00
3  95,0° C for 0:15
4  60,0° C for 0:50
5  GOTO 3, 44 more times
6  35,0° C for 1:00
7  75,0° C for 2:00
   Slow Ramp Rate to 0,1°C per second
8  31,0° C for 5:00
9  Melt Curve 31,0 to 74,0° C, increment 0,5° C
   for 0:05 + Plate Read
   END
    
```

The image reproduces the thermal profile as it appears in the instrument program.

- Click OK. Save the new protocol.

⇒ PLATE SET UP

Select "plate" in the "Run setup" window: it is possible to create a new plate or select/modify a pre-existing one.

Creating a new plate

- "File → New Plate"
- The "Plate Editor – new" window will be open
- In "Scan Mode" select "All channels"
- Click "Select Fluorophores" and select FAM, HEX, ROX, Cy5, Cy5.5
- Select the plate wells where the samples will be placed.
- Select in "Sample Type → Unknown"
- Select "Target Name" and check the "Load" box to insert the fluorophores. Name the targets **exactly** as shown in Table 1

target	Fluorophore
C677T	FAM
FV Leiden	HEX
A1298C e FII	ROX
PAI-1 4G/5G	Cy5
FV HR2	Cy5.5

Table 1. Target Name settings

- Select one plate position at a time and enter the sample name in the "Sample Name" field; then check the "Load" box
- Click "OK". Save the new plate (Figure 1)

Figure 1. Set-up of the plate

- In case of insertion of Positive Controls select in *Sample Type* → *Positive Control*.

PROTOCOL OR EXISTING PLATE SELECTION AND MODIFICATION

To import a previously saved thermal profile (*.prcl) or plate (*.pltd) from the PC, select "*File* → *Open Protocol*" or "*File* → *Open Plate*" respectively. It is possible to modify the recalled protocol or plate and save the performed changes.

STARTING RUN

- Select "*Start Run*" in the "*Run Setup*" window. The instrument is ready to start the run.
- Prepare the plate/strip with samples and mix.
- Open the lid, place the plate/strip in the instrument and close it.
- Select "*Start Run*" to start.
- Do not change the default name by which the run is saved by the CFX. The information contained is necessary for the traceability of the run as it reports the date and start time of analysis.

⇒ DATA ANALYSIS

- At the end of the run the "Data Analysis" window will open automatically.
- Select "Melt Curve": a new window shows the displayed charts of the run. The chart on the left shows the raw data of the melting curve (RFU), the chart on the right shows the $-d(\text{RFU})/dT$ analysis with the peaks of the melting curves.
- Select "Peak Type → Positive"
- Set the correct threshold manually for each individual factor to eliminate background noise and/or any non specific peaks (reff. 2a and 2b Figueres).
- Select "Melt Curve Data" and analyze the melting peaks obtained for each sample using the following Table.

Target Factor	Fluorophore	T _m 1			T _m 2			T _m 1+ T _m 2
		[°C]	[°C]	Genotype	[°C]	[°C]	Genotype	Genotype
FII G20210A	ROX	62	± 2	Wild type	68	± 2	Homozygous mutated	Heterozygous
FV Leiden G1691A	HEX	44	± 2	Wild type	53	± 2	Homozygous mutated	Heterozygous
FV H1299R	CY5.5	46	± 2	Wild type	59	± 2	Homozygous mutated	Heterozygous
MTHFR C677T	FAM	59	± 2	Wild type	50	± 2	Homozygous mutated	Heterozygous
MTHFR A1298C	ROX	45	± 2	Wild type	54	± 2	Homozygous mutated	Heterozygous
PAI-1 (4G/5G)	CY5	56	± 2	Homozygous 4G	62	± 2	Homozygous 5G	Heterozygous 4G/5G

Table 2. Interpretative table for sample genotyping

WARNING

1. To change the Threshold manually, click on the Melt Peak chart and move the threshold to remove background signal.

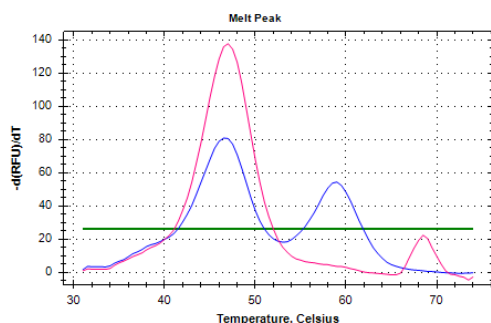
Figure 2a. Graph for Factor V H1299R with threshold set manually to eliminate background signal.

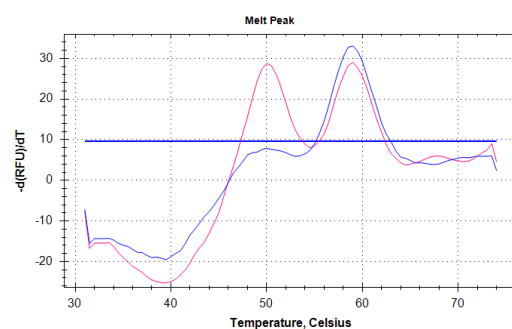
Figure 2b. Graph for MTHFR C677T with threshold set manually to eliminate background signal.

2. In case of signals in NTC sample, consider the results as below:

<i>Signal</i>	<i>Interpretation</i>	<i>Result</i>
Peaks detected in number of channels < 5	Artificial signal	valid NTC, it is possible to proceed with the analysis of the samples
Peaks detected in all channels	Contamination	Not valid NTC, it is recommended to repeat the run

Do not consider peaks out of the temperature range.

For automatic interpretation of results and report generation, use the **Real Gene** program following the NLM code **DO022** user manual.

The Positive Control inclusion in the run is optional. Non-compliant CP results do not invalidate the run.

⇒ PERFORMANCE CHARACTERISTICS

Analytical Sensitivity

The analytical sensitivity of the “CVD6 Multiplex Real Time” kit, expressed as the minimum quantity of the target that can be detected (probability >99%), is 0.9 ng/μl of DNA (equivalent to 3.6 ng/reaction) and was evaluated by testing serial DNA dilutions of three different samples, extracted with AA1001 NLM code and run following instructions for use.

Diagnostic Sensitivity

The diagnostic sensitivity of the “CVD6 Multiplex Real Time” kit was assessed by analyzing 69 samples with a known genotype. All samples were correctly genotyped, resulting in a diagnostic sensitivity of 100%.

Diagnostic Specificity

The specificity of the “CVD6 Multiplex Real Time” kit was determined by analyzing 69 samples with a known genotype. No false positive results were obtained. In agreement with these data the specificity of the kit is 100%.

Reproducibility

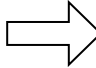
Intra-assay reproducibility

The intra-assay reproducibility was assessed by analyzing 6 different samples in 3 replicates in the same run.

Inter-assay reproducibility

The inter-assay reproducibility was assessed by analyzing 5 different samples in 3 different run, performed by different operators.

In agreement with the obtained results, reproducibility is 100%.

					
Lot Number	Catalogue number	Temperature Limitation	Use by	Keep away from sunlight	Consult Instructions for use
					
European Conformity	Manufacturer	<i>in vitro</i> diagnostic device	Sufficient for	Positive Control	Modified text compared with the previous version