**REF** AA842/50

**50 TEST**

**CND** W0105010117

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## CHLAMYDIA TRACHOMATIS REAL TIME 2.0

**CE**  
0459

**IVD**

NUCLEAR LASER MEDICINE S.r.l.  
UFFICI OPERATIVI: Viale delle Industrie, 3 – 20090 SETTALA MI (Italy)  
Tel (+39) 02/95. 24. 51 - Fax (+39) 02/95. 24. 52. 37  
SITO INTERNET: [www.nlm.it](http://www.nlm.it) – E-MAIL: [segreteria@nlm.it](mailto:segreteria@nlm.it)

## ⇒ UTILIZZO

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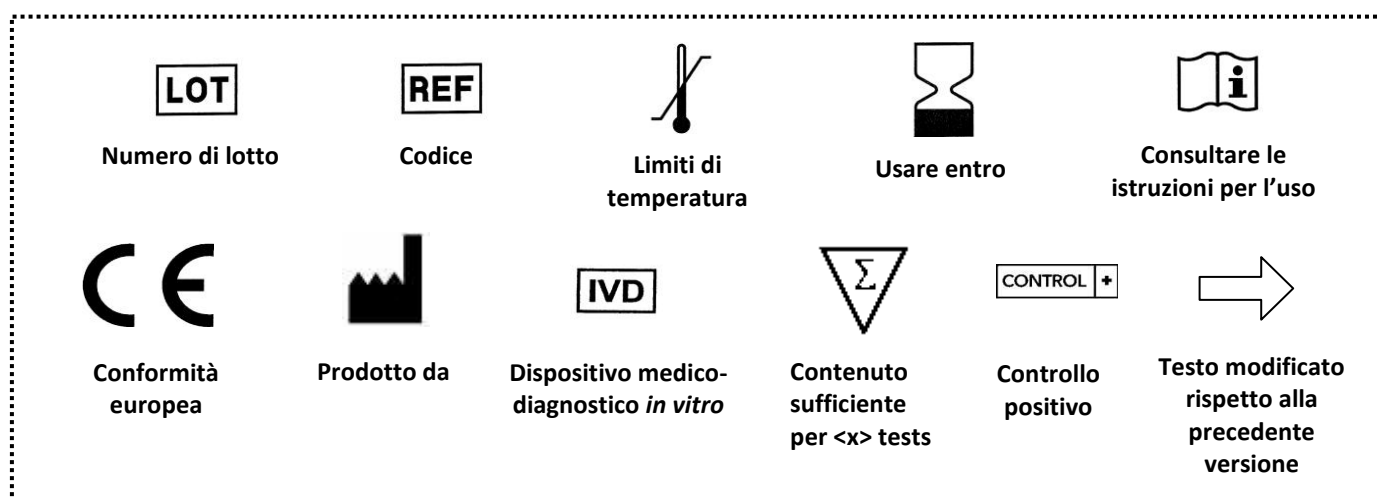
## ⇒ INTENDED USE

CHLAMIDIA TRACHOMATIS REAL TIME 2.0 provides reagents for qualitative determination of *Chlamydia trachomatis* (CT) DNA and simultaneous discrimination of serotype L by Real Time PCR in human female and male samples (urine, seminal fluid and endocervical, vaginal, urethral, anorectal, pharyngeal and eye swabs).

The device has been validated with manual DNA extraction column system (NLM code AA1001) and with automatic extraction systems: MagCore (NLM code AA1185), Janus Perkin Elmer (NLM code AA1318) or QIA Symphony (NLM code AA1440/96).

Amplification of extracted DNA has been validated with the following PCR Real Time Instruments: CFX (BioRad), Rotor-Gene Q (Qiagen) and 7500 Fast (Applied Biosystem).

To detect the presence of *C. trachomatis* DNA, an instrument with the specific channels for FAM and JOE dyes is sufficient; for simultaneous discrimination of the L serotype, an instrument with also the specific channel for CY5 dye is required.

The device provides also the enzyme Uracil-N-glycosidase (UNG) to prevent contamination and false positive results.

Amplification enzyme assigned: **orange lid** NLM code BA092, followed by size indication, which could change depending on enzyme quantity supplied (i.e.: BA092/50, BA092/60, etc).

The device is to be considered for professional use only.

## INTRODUCTION

*Chlamydia trachomatis* (CT) is a gram-negative obligate intracellular parasite of eukaryotic cells. CT has a 1000 kbp genomic DNA and a 7,5 Kbp extrachromosomal cryptic plasmid (7-10 copies per bacterium) which, in some strains, could be lost or partially deleted (nvCT)<sup>1,2</sup>.

Most *C. trachomatis* infections are asymptomatic in both males and females; however, an incorrect diagnosis may lead to irreversible deterioration in the development of the disease: for this reason it is important to make a timely and accurate diagnosis of infection<sup>3,4</sup>.

In particular, CT includes numerous serotypes, determined by the membrane protein MOMP (Major Outer Membrane Protein) encoded by the gene *omp1*, which are associated with different clinical pictures<sup>5</sup>.

The serotypes A to C are responsible for ocular conjunctivitis and trachoma which can lead to blindness, serotypes D to K are associated with genital disease, with the risk of causing infertility and even conjunctivitis and pneumonia in infants of infected mothers. Serotypes L1, L2 and L3 represent the etiological agent of lymphogranuloma venereum.

## REFERENCES

- 1) Stephens RS et al. Science 23 Oct 1998:Vol. 282, Issue 5389, pp. 754-759
- 2) Magnus Unemo et al. Microbiology (2010), 156,1394–1404
- 3) WHO guidelines for *Chlamydia trachomatis*, 2016
- 4) 2015 European guideline on the management of *Chlamydia trachomatis* infections
- 5) Jurstrand M et al, Journal of clinical microbiology, Nov. 2001, p. 3915-3919

## PRINCIPLES OF THE PROCEDURE

CHLAMIDIA TRACHOMATIS REAL TIME 2.0 is based on two processes:

1. Bacterial DNA extraction
2. Amplification and detection of target sequence using Real Time PCR

The Internal Control (Coagulation Factor V) is endogenous and therefore is extracted together with the sample in order to monitor the entire procedure. A positive control (PC) is also included in order to verify the correct performance of the analysis.

### 1. DNA extraction

CT DNA and Internal Control (IC) can be extracted from various human samples (urine, swabs and seminal fluid), using manual DNA extraction column system (NLM code AA1001) or automatic extraction systems: MagCore (NLM code AA1185), Janus - Perkin Elmer (cod. AA1318 NLM) or QIASymphony (cod. NLM AA1440/96).

### 2. PCR and Detection

#### Amplification target

Amplification and detection of the 16S rRNA gene, located on the CT genome, and the cryptic plasmid allow to determine the presence of bacterial DNA, while amplification of a portion of the gene *omp1* (localized in the bacterial genome and coding for MOMP protein) and detection by specific probes for serotypes L1, L2 and L3 allow to discriminate the serotype.

Primers that amplify the cryptic plasmid were designed in order to avoid the 377 bp deletion identified in a variant strain (nvCT).

The IC (Factor V) has an internal sequence similar for length and base composition to CT target sequence and specific primers and probe that differentiate the ICs from CT.

The positive control (PC) consists of synthetic DNA of cloned *16S rRNA* gene and *omp1* gene (serotype L).

#### Amplification

Following extraction step, DNA (CT and IC) is amplified by PCR, using appropriate enzymes and buffer conditions.

#### Detection

The test is based on the Real Time PCR, a technique that allows to monitor in real time the amplification of the samples of interest, using probes doubly labeled with a donor fluorophore and a quencher ("dual-labeled probe").

The probes are labelled with different fluorophores donors in order to distinguish the different targets:

Fluorophore	Target
FAM	16S and cryptic plasmid
CY5	<i>omp1</i> gene of serotypes L1, L2 and L3
JOE	Factor V

In presence of a light source, when the probes are intact and donor and quencher are close to each other, the fluorescence emitted by the donor fluorophore is absorbed by the proximal quencher (figure 1a). During the amplification each probe matches to its specific sequence and it is target of 5' → 3' nuclease activity of the DNA polymerase. When donor and acceptor fluorophores are separated, the donor fluorescence can be detected at its specific wavelength: in this way, the amplification of the bacterial DNA and the IC can be monitored as the reaction proceeds (figure 1b).

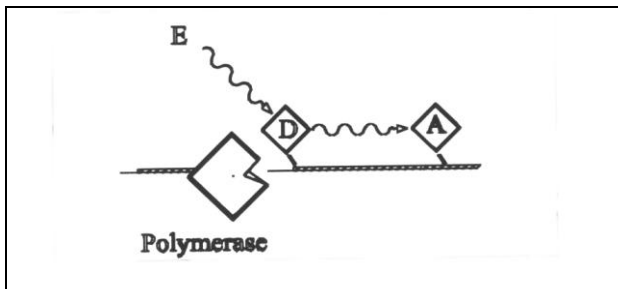
Fig.1a) Energy (E) emitted by the donor (D) fluorophore is absorbed by the acceptor (A) fluorophore.

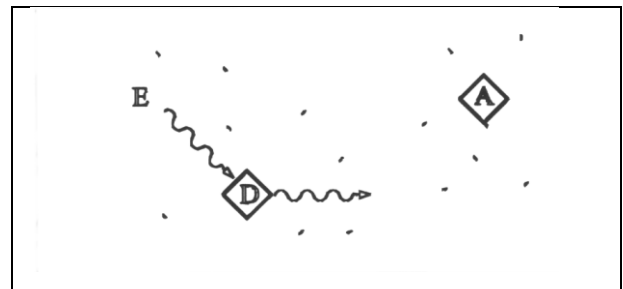
Fig.1b) The polymerase exonuclease activity separates the D fluorophore from the A fluorophore by hydrolysis resulting in an increase in fluorescent signal

Real Time tools include in a single instrument a thermalcycler for target amplification and a fluorometer for the detection of fluorescence during the cycling process. The computer connected to the system collects the fluorescent data, displayed in a graph through a specific software (figure 2a). After the raw data collection, analysis is carried out. Raw data is normalized to correct the background signal; then a threshold level can be set: this is the level at which fluorescence data is analyzed (figure 2b).

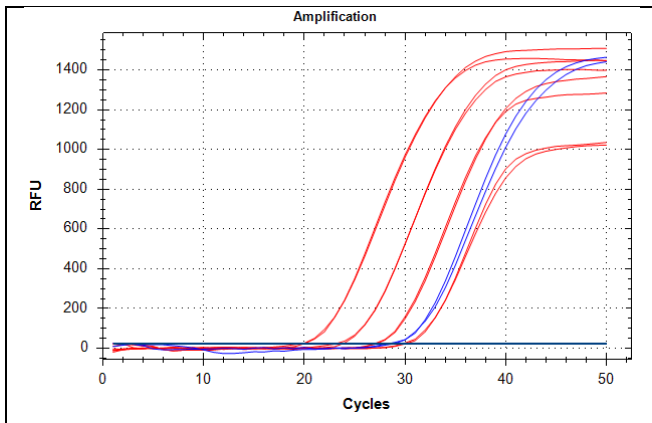
Figure 2a: raw data is plotted on a graph of fluorescence versus cycle number.

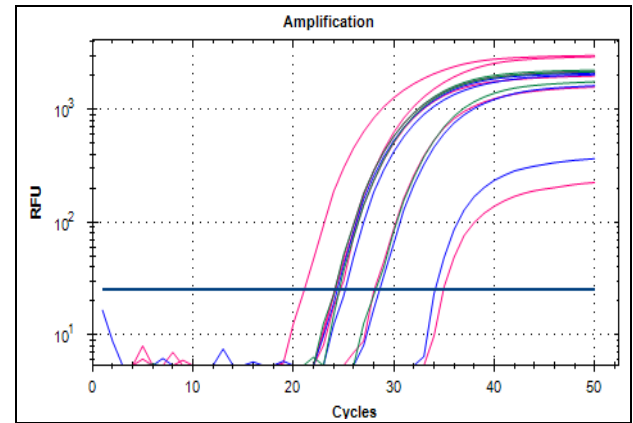
Figure 2b: the normalized data is plotted on a log scale graph of fluorescence versus cycle number.

Ct-value (threshold cycle) is the number of cycles necessary for a sample to reach the threshold level and it is related to the initial amount of target DNA: the higher the initial titer of the target, the earlier the fluorescence signal reaches the threshold level

#### PCR product contamination Control

PCR amplification is performed in the presence of deoxyuridine triphosphate (dUTP) to generate amplification products that can be selectively degraded by treatment with uracil-N-glycosylase (UNG) and heat.

UNG cleaves the glycosidic bond between the uracil base and the deoxyribose ring, creating an abasic site that is susceptible to heat. A + 25 °C incubation prior to the amplification step removes uracil bases from any contaminating amplification products present in the reaction mixture. The heat-labile UNG enzyme is inactivated completely and irreversibly when the temperature is increased during the RT incubation, preventing removal of uracil from the newly synthesized cDNA.

## PRODUCT COMPOSITION (Store at -25/-15°C)

<b>Mix 1</b> Tris HCl KCl MgCl <sub>2</sub> dNTPs	<b>1 x 600 µl</b>
<b>Mix 2</b> Primers and probes Tris-EDTA	<b>1 x 330 µl</b>
<b>Mix 3</b> PCR Enzymes	<b>1 x 20 µl</b>
<b>RNase, DNase, Protease free water</b>	<b>1 x 250 µl</b>
<b>Positive Control</b> CT Template Positive Control is supplied lyophilized and must be resuspended in 30 µl of RNase, DNase and Protease free water before use in Real Time-PCR. The use of the PC is optional.	<b>5 vials</b>

## STABILITY AND STORAGE

- All the reagents are stable up to the expiry date indicated on the label when stored at the correct temperature (-25/-15°C).
- Thaw reagents on ice or at +2/+8°C.
- **Mix** are stable until the expiration date if kept at -25/-15°C. These reagents can be thawed and frozen max 5 times. If you plan a greater number of uses, aliquot the mix and to each tube apply the same limit.
- **Mix 2** contains the FAM, JOE and CY5 fluorophores that are photosensitive: avoid prolonged exposure to light.
- **Master Mix** should be used immediately after preparation; after it is dispensed into PCR tubes, the remaining Master Mix should be discarded. Avoid prolonged exposure to light.
- **Positive Control** is a DNA template, supplied already aliquoted and lyophilized: it is recommended to re-suspend such reagent with RNase, DNase and Protease free water (supplied) before use, after samples dispensing in the respective tubes in order to avoid contamination.

## PRECAUTIONS

- Only professional and opportunely trained personnel should use this kit. Handle this product according to established Good Laboratory Practices and universal precautions; wear personal protective apparel.
- All disposable items (tips and tubes) must be DNase, RNase and DNA free. Use aerosol-resistant pipette tips to avoid pipettes contamination. Use a new tip every time a volume is dispensed.
- Discard used materials as bio hazardous waste.
- Do not eat, drink, smoke or apply cosmetics in areas where reagents or specimens are handled.
- If skin or mucous membrane exposure occurs, immediately wash the area with copious amount of water. Seek medical advice.
- Do not use components beyond the expiration date
- Do not mix reagents from different lots.
- Due to the high sensibility of the test, particular attention should be paid attention to cross-contaminations. It's recommended to perform the assay in three different areas:
  - Area 1: pre-PCR (samples handling and extraction)
  - Area 2: Master Mix preparation.
  - Area 3: post-PCR (Real Time PCR)
- It's recommended to clean the workstation with chlorine bleach at the end of the procedure
- Don't use the device if the box is damaged; contact the supplier.
- It is advisable to have constant and uniform laboratory temperature, avoid to place the instruments near heating/cooling sources that may compromise the correct work.

## MATERIALS REQUIRED BUT NOT SUPPLIED

### AREA 1

DNA Extraction Kit

Vertical downflow airbox

Dedicated adjustable volume pipettes set and aerosol barrier tips

2 ml and 1,5 ml DNase, RNase free tubes

### AREA 2

Vertical downflow airbox

Dedicated adjustable volume pipettes set

Aerosol-resistant pipettes tips

⇒ 0,2 ml PCR tubes, strips or plates designed for Real Time PCR (for Bio-Rad CFX it is recommended to use transparent plastic consumables)

DNA polymerase with **orange cap**

### AREA 3

Real Time PCR instrument

## COLLECTION, STORAGE AND PREPARATION

### ⇒ Collection and storage

- swabs: must be prepared and stored according to laboratory specifications. Unless otherwise indicated, it is suggested to keep at +2 / +8 °C for a maximum of three days and not freeze in order to avoid cell lysis and loss of bacterial DNA titer.
- urine samples: first void urine must be collected, in containers without preservatives according to the indication of the laboratory, transported at room temperatures (+18 / +25 °C) for max four hours or stored at +2 / +8 °C for max three days. If possible, avoid freezing urine samples. Freezing may cause precipitation of inhibitors and the lysis of the cells and the loss of bacterial DNA.
- seminal fluid: if the samples are not used immediately for DNA extraction, store them at -20°C.

### ⇒ Preparation

- swabs: vortex vigorously the specimen and proceed with the DNA extraction. Centrifuge samples containing a small number of cells (leucocytes, epithelial cells) at 12000-14000 rpm for 5 minutes. Remove the supernatant, leaving a amount of volume depending on extraction system protocol. Vortex for 7-10 sec. and proceed with the DNA extraction.
- Urine samples: centrifuge 15mL of urine at 6000 rpm for 15 min. Discard the supernatant and resuspend the pellet in 1400 µl of PBS.
- Seminal fluid: resuspend in 1 volume of PBS.

### ⇒ BACTERIAL DNA EXTRACTION

For preparation, handling and disposal of reagents refer to the specific instructions for use of the extraction system.

Add elution volume as follows:

extraction system	starting volume	elution volume
Manual DNA extraction column system (NLM code <b>AA1001</b> ).	refer to the specific instructions	100 µl
Automatic extraction MagCore (NLM code <b>AA1185</b> ).	400 µl	60 µl
Automatic extraction with Janus - Perkin Elmer (NLM code <b>AA1318 – AA1318 Microbiologia protocol</b> ).	refer to the specific instructions	refer to the specific instructions
Automatic extraction with Qiagen - QIA Symphony (NLM code <b>AA1440/96 – Cell free 500 protocol</b> ).	refer to the specific instructions	85 µl

Purified DNA can be stored at +2/+8° C if immediately used, otherwise store at -25/-15° C. It is recommended to thaw at +2/+8° C.

## REAL TIME PCR

### Warning

- Avoid using bacterial DNA not correctly stored in order to prevent bad assay results.
- Use dedicated Real Time PCR tubes only.
- We recommend to set the thermal profile before the amplification mix preparation.
- It is recommended to validate the entire analysis procedure of each session, DNA extraction and amplification, using a negative control and a positive control (positive sample already tested or synthetic PC supplied).

⇒ Prepare the master mix as follows:

Reagents	Volume/sample
Mix 1	9,00 µl
Mix 2	5,45 µl
DNA polymerase (5U/µl) <b>orange cap</b>	0,30 µl
Mix 3	0,25 µl

- Resuspend well the mix before use.
- Prepare the Master Mix for the number of extracted samples + 1 Positive Control (optional) + 1 more volume (for no ≤ 10) or + 2 more volumes (for no >10).
- Mix gently and dispense **15 µl** of Master Mix in the previously marked Real Time PCR tubes, strip or plate.
- Add **10 µl** of extracted DNA to the respective tubes.
- Add **30 µl** of DNase, RNase and Proteinase free water to the Positive Control and resuspend well; then transfer **10 µl** of solution into tube with the Master Mix.
- Discard unused portion of Master Mix.

⇒ **WARNING:** dispense and pipette mix and samples very carefully, in order to avoid creating air bubbles. To avoid the risk of contamination, add PC after dispensing the samples. If it is possible, centrifuge the plate/strip before placing it into the Real Time instrument.

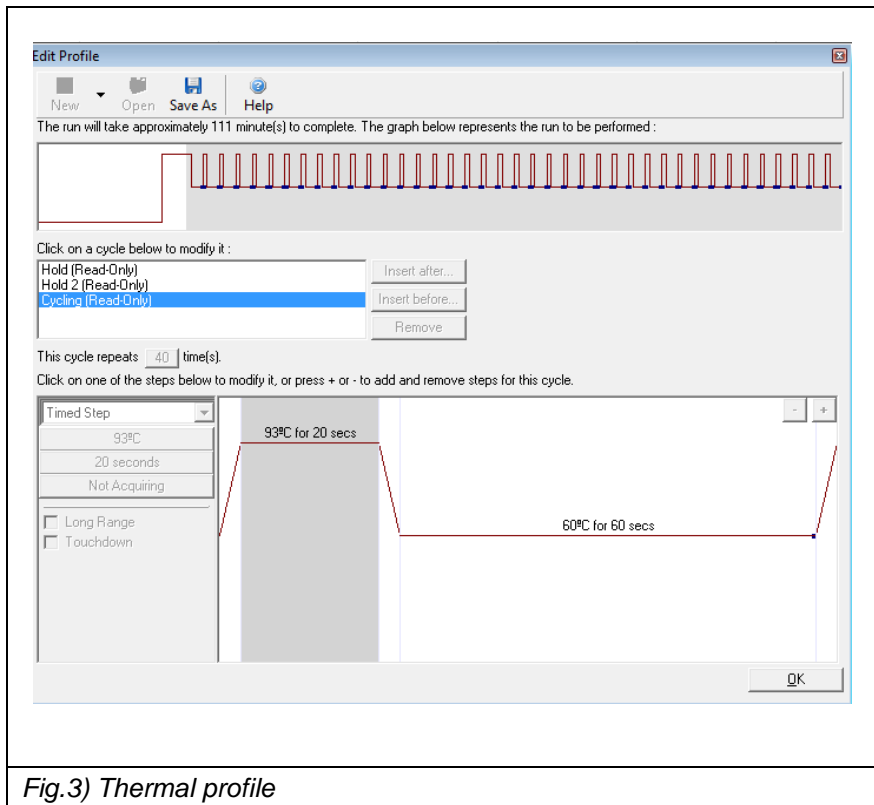
## THERMAL PROFILE SETTING AND ANALISYS

⇒ **ROTORGENE Q (Rotor-Gene Q Series Software 2.x)**

- Use the software function “Edit Samples” in order to insert samples position and sample type (unknown, positive control, negative control).
- In the “Setting” section set the reaction volume (25 µl) and the used rotor (36/72 positions). Check in the “Channels” window if **Green, Yellow** and **Red** dyes are already present, otherwise click on “Create new” in order to insert it.
- In the “View” section → “gain optimization” set autocalibration temperature to 60 °C for channels above (**Min reading 5 FI; Max reading 10 FI**) and activate the “Perform optimization before first acquisition” checkbox.
- In the “Profile” section set the following thermal profile:

Cycle	Cycle Point
Decontamination	<b>25°C</b> , 10 min
Denature	<b>93°C</b> , 2 min
Cycling: 40 repeats	Step 1: <b>93°C</b> , 20 sec
	Step 2: <b>60°C</b> , 60 sec; acquiring to Green, Yellow and Red

- Place tubes in the RotorGene and close the lid.
- Run the assay by clicking the "Start" button.


### Interpretation of results

At the end of the experiment, proceed with the analysis of data.

Analyze each channel separately and proceed as described below:

- Select "Analysis", "Quantitation", then select the channel of interest: "Cycling A.Green", "Cycling A.Yellow" (fig. 4) or "Cycling A.Red (fig. 5), "Show".

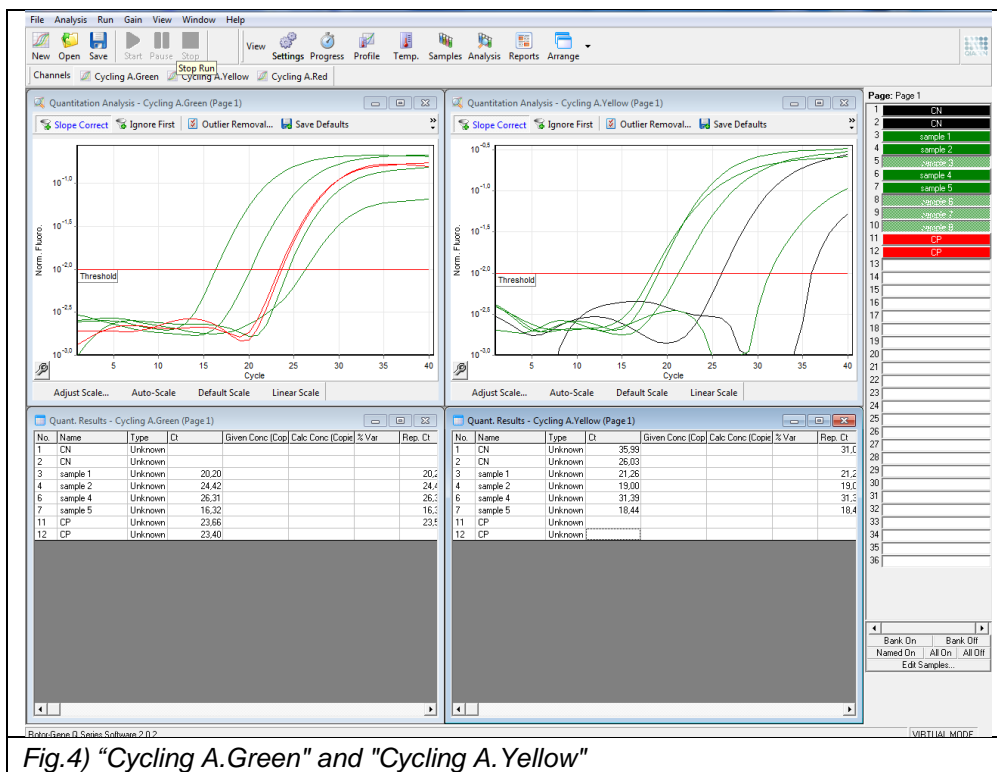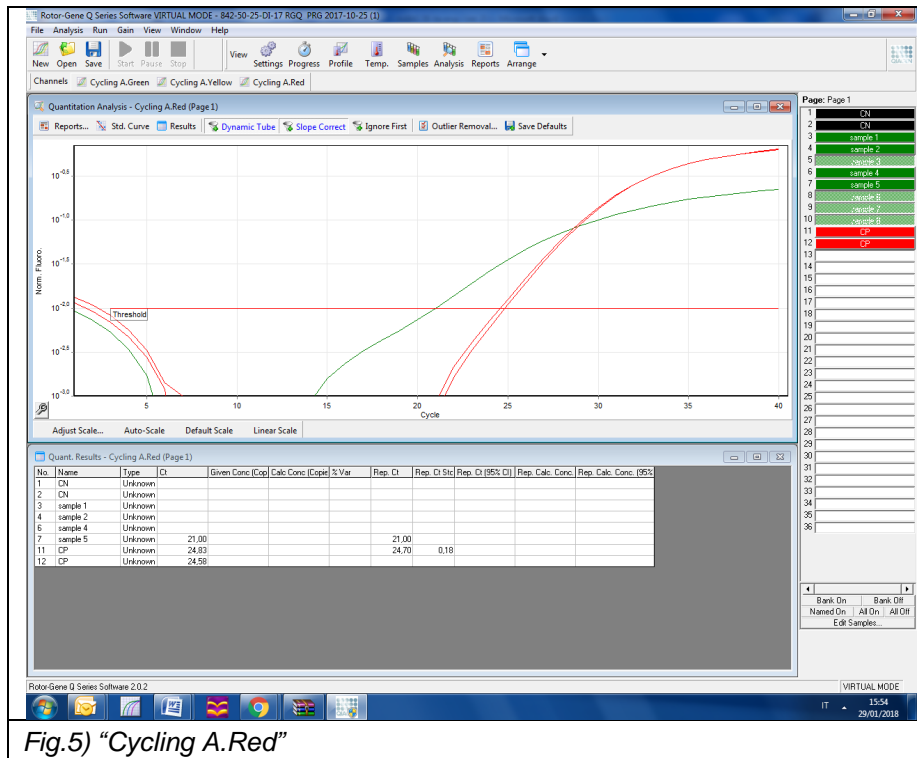
Fig.5) "Cycling A.Red"

- In the analysis window with the chart, select "Dynamic Tube" and "Slope Correct"; if necessary, in the "Outlier Removal" box, set from 10 to 20%.
- Enter the "Threshold" value 0.01 and, if necessary, eliminate the first 10-12 cycles: the intersection between the threshold line and the curve of the sample represents the cycle threshold (Ct) (fig. 6).

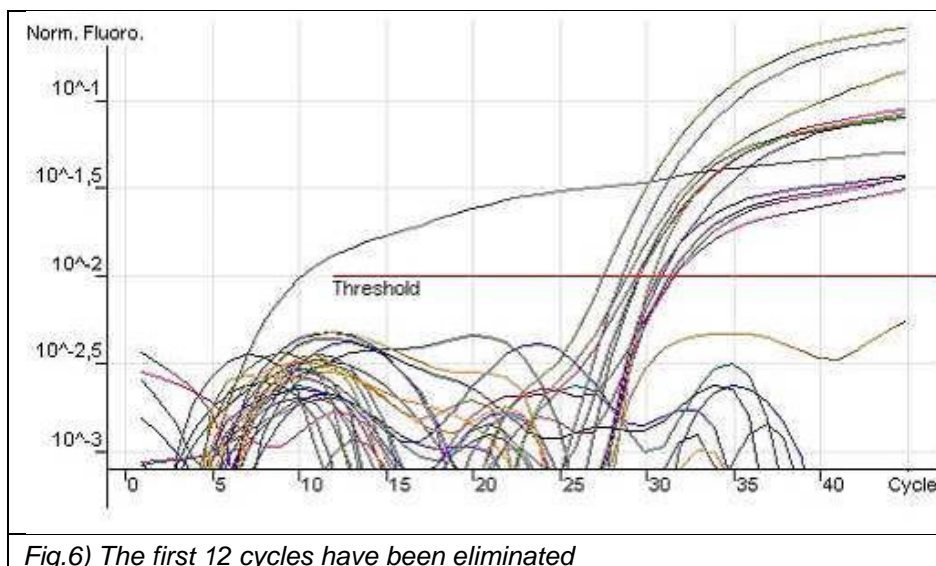
Fig.6) The first 12 cycles have been eliminated

## CFX 96 (CFX Manager Software v1.6 and 3.x)

- Open the “Bio-Rad CFX Manager Software”.
- Click on “*Create new run*”, select the model of instrument used and click “OK”.
- The “*Run Setup*” window will open.

- **Thermal profile**

It is possible to create a new protocol, select/modify an existing one or repeat a run.

### Create a new protocol

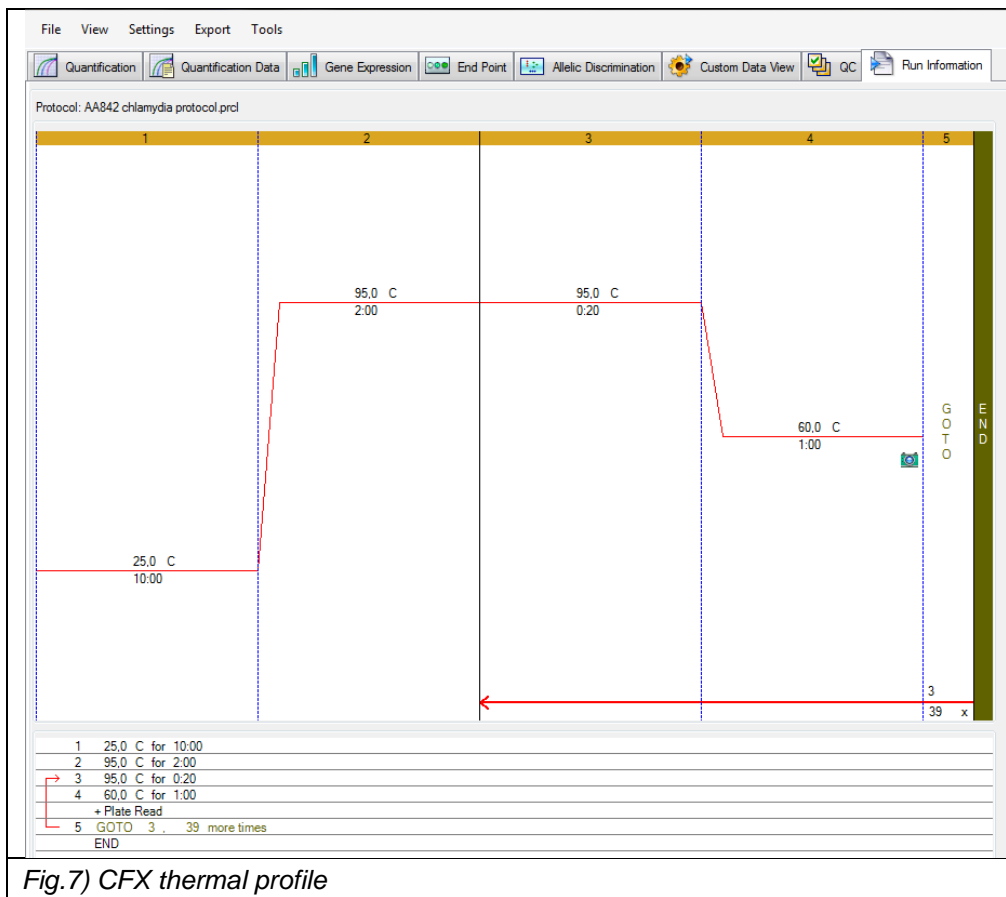
- “*Protocol → create New*”.
- “*Protocol Editor – new*”.
- Enter the reaction volume in the “*Sample Volume*” box (25 µl)
- Add the following thermal profile:

Step	Settings
Decontamination	<b>25°C</b> , 10 min
Denature	<b>95°C</b> , 2 min
Amplification: 40 cycle	Step 1: <b>95°C</b> , 20 sec
	Step 2: <b>60°C</b> , 60 sec; acquiring to annealing step (plate read)

- To add a step, click “Insert Step” on the left side of the window. The step will be added in the next position to the selected step.
- It is possible to change the temperature and duration of the steps in the chart or in the text written below, by double clicking on them.
- In the step of annealing, check that the symbol of the camera is present in the chart and the word “+ Plate Read” is present in the text. If not, add them by clicking “Add Plate Read to Step”.
- Click OK. Save the new protocol.

### Select/modify existing protocol

- It is possible to import a temperature profile from the saved profiles, stored on PC (\*.prcl): to import an existing protocol click “*Select Existing*” in “*Protocol*”, “*Run Setup*”.
- To modify an existing plate click “*Select Existing*” and “*Edit Select*”.
- Proceed as explained above.

- **Plate set up**

- Open “plate” in “Run setup” window.
- It is possible to create a new plate or select/modify an existing one.

⇒ **Create a new plate**

- “Plate → create New”.
- The “Plate Editor – new” window will open.
- Set the plate type by selecting “Settings → View/Edit Plate → Settings → Plate Type” and select “BR Clear”.
- Select “Scan Mode → All channels”.
- Click “Select Fluorophores” and check FAM, JOE and CY5 box (if JOE is not present, select the corresponding HEX fluorophore).
- Select wells.
- Choose “Sample Type → Unknown, Negative or Positive Control”.
- Type into “Target Name” the name of target (FAM for CT, JOE for IC and CY5 for Ser L) and click “Load” check boxes to insert fluorophores.
- Type in “Sample Name” the name of the sample, then click “Load”.
- Click OK. Save the new plate.

## Select/modify existing plate

- It is possible to import a plate from the saved profiles, stored on PC (\*.prcl): to import an existing plate click “*Select Existing*” in *Plate* section.
- To modify an existing plate click “*Select Existing*” and “*Edit Select*”.
- Proceed as explained above.

### • Start of the experiment

- Select *Run Setup*, *Start Run*.
- The instrument is now ready to work. Prepare plates / strips with samples and mix.
- Open the lid by clicking the button “*Open Lid*” and place the plate / strip in the instrument; close the lid by clicking the “*Close Lid*” button.
- Click “*Start Run*” to start the session.
- It opens the *Run Details* window. Move to *Real Time Status* page to monitor the progress of the raw data of the experiment.

### • Data Analysis

- At the end of the experiment “*Data Analysis*” window opens automatically.
- In the upper left side select “*Settings*” and set:
  - *Cq determination mode: single threshold*
  - *Settings – analysis mode (ver. 1.6) / Baseline Settings (ver. 3.x): Baseline subtracted curve fit (CFX Manager ver. 1.6 or later).*
  - Select the three different channels separately. For each channel select: “*Settings, Baseline threshold*”:
    - *Single Threshold-User defined-set: 100*
    - only for samples that show a signal artifact, select “*Baseline cycles*” → select all the wells and set “*Baseline Begin*”: 5 and “*Baseline End*”: 15 (fig. 8).

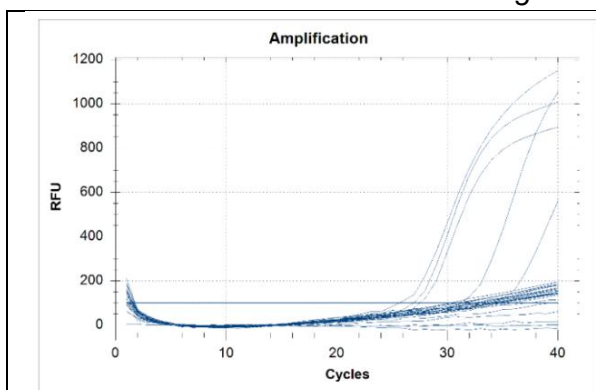
Fig. 8a) Es1: Baseline methode: Auto Calculated

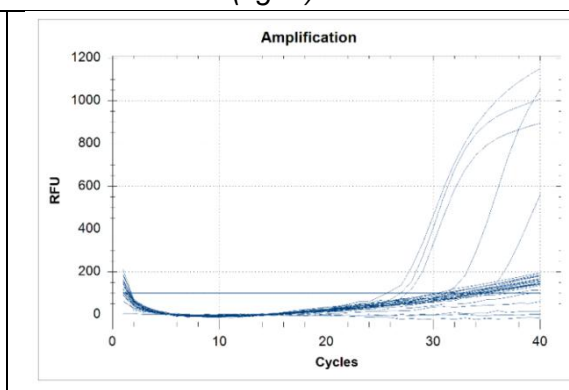
Fig. 8b) Es1: Baseline methode: User Defined

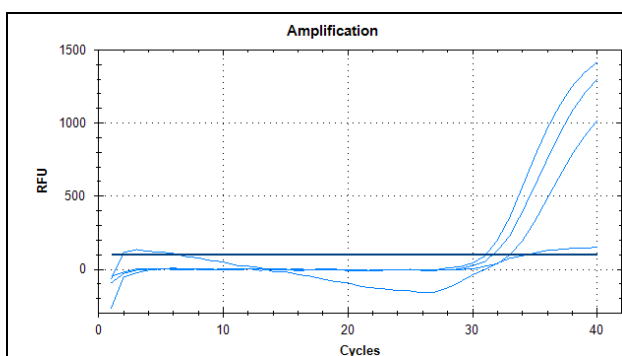
Fig. 8c) Es2: Baseline methode: Auto Calculated

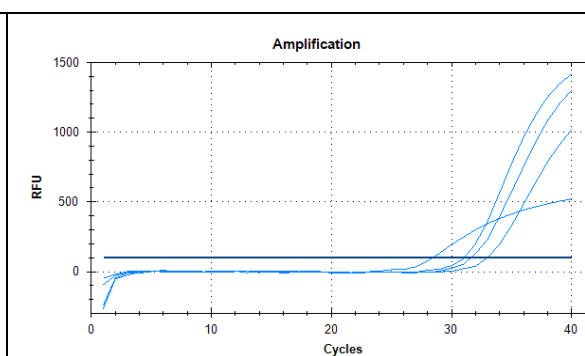
Fig. 8d) Es2: Baseline methode: User Defined

## ABI 7500 FAST (7500 Software 2.x)

Open the corresponding software and click “*Advanced Setup*” in the left side of the screen: a new window opens.

- In the “*Setup*” menu, on the left side, select *Experiment Properties*, type the experiment name, and select the “*Fast 7500 (96 wells)*”, “*Quantitation Standard Curve*”, “*TaqMan Reagents*” and “*standards*” (2 hours to complete a run).

### • Plate set up

- In the “*Setup*” menu, on the left side, select “*Plate Setup*”.
- Select “*define targets and samples*” at the top.
- In the left side of the page (“*define targets*”) select “*add new target*” and enter the name of the target; from the dropdown menu, select “*FAM*” as a reporter, “*NFQ-MGB*” (“*non-fluorescence sequencer-minor groove binder*”) as a quencher and choose a color. Repeat the operation, selecting “*JOE*” and “*Cy5*” as a reporter.
- At the right side of the page (“*define samples*”) select “*add new sample*” for each sample that has to be analyzed and enter the “*name of the sample*”.
- Select “*Assign targets and samples*” at the top.
- Select the wells used in the plate for the session and match the fluorophore, clicking the target on the left side (*assign*). Also indicate in “*Task column*” the type of sample analyzed in that position: eg. *Positive control*, *Negative control* or *Unknown* (for samples).
- Select “*NONE*” into the “*select the dye to use as the passive reference*” box.

### • Thermal profile (fig. 9)

- Select (on the left) “*Run Method*”.
- Select (at the top) “*Graphical View*”.
- Insert 25 *uL* as *Reaction volume* per well.
- Set up the specific thermal profile.
- From the dropdown menu, select “*Add stage Holging or Cycling*” to add stage, while in order to add a step in the same stage, use “*Add Step*” menu.
- Ramp: all ramps must be 100% as default.
- Set the acquisition of the fluorescence signal only in the annealing step in PCR cycles.
- Leave unchecked “*Enable Autodelta*”.

Step		Settings	
Holding Stage	Step 1	25°C - 10 min	Ramps 100%
Holding Stage	Step 1	95°C - 2 min	
Cycling Stage 40 cicli	Step 1	95°C - 20 sec	
	Step 2	60°C - 1 min <b>Acquired</b>	

- Verify in the *Run window* and check that the *Instrument Status* appears connected: the instrument is ready to work.

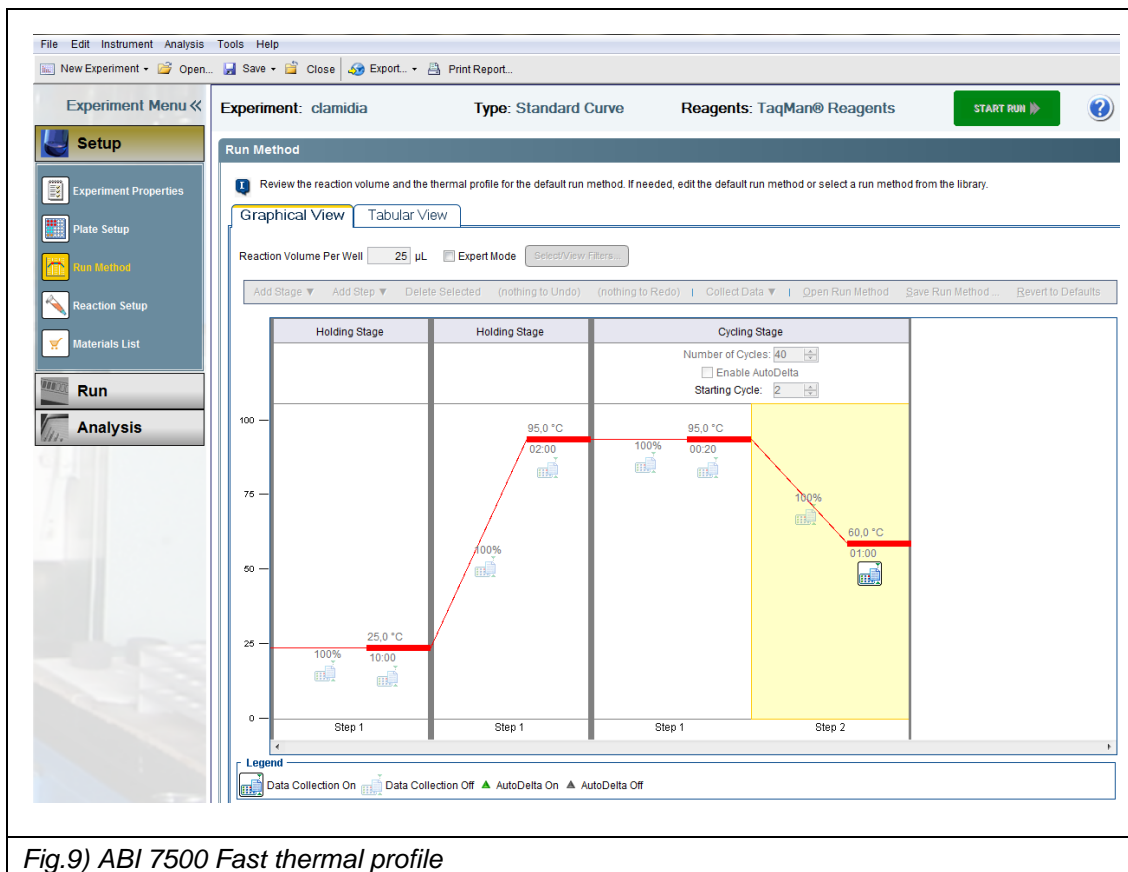
Fig.9) ABI 7500 Fast thermal profile

- **Start of the experiment**

- Prepare the plate with samples and mix, seal it with the specific adhesive film taking care not to create air bubbles.

Centrifuge the plate a few seconds to remove any bubbles in the wells that may distort the reading.

- Save the experiment session.
- Press “Start” to begin.
- To save the template, select “File, Save as template”. In following experiments it will be possible to retrieve and modify the template by selecting “File, New experiment, From template”.

- **Data analysis (fig. 10)**

- Select “Analysis” on the left of the window.
- To view amplification curves, select “Amplification plot” and set as plot type “dRn vs Cycle”, “Linear” graph type and the target to be viewed.
- To change the *Baseline and Threshold*, select in the top panel “Analysis”, “Analysis Settings” and open the “Ct Settings” window, then select the “Target” to be edited:

- de-select “Use Default Settings”.
- set the following parameters:

<b>FAM</b>	100000
<b>JOE</b>	50000
<b>CY5</b>	50000
<b>Autobaseline selection</b>	<b>NO (Start=3; End=15)</b>

- select “Apply Analysis Settings”.

- Always click "Reanalyse".
- In the "View Well Table" it is possible to display Ct and titers of each sample.

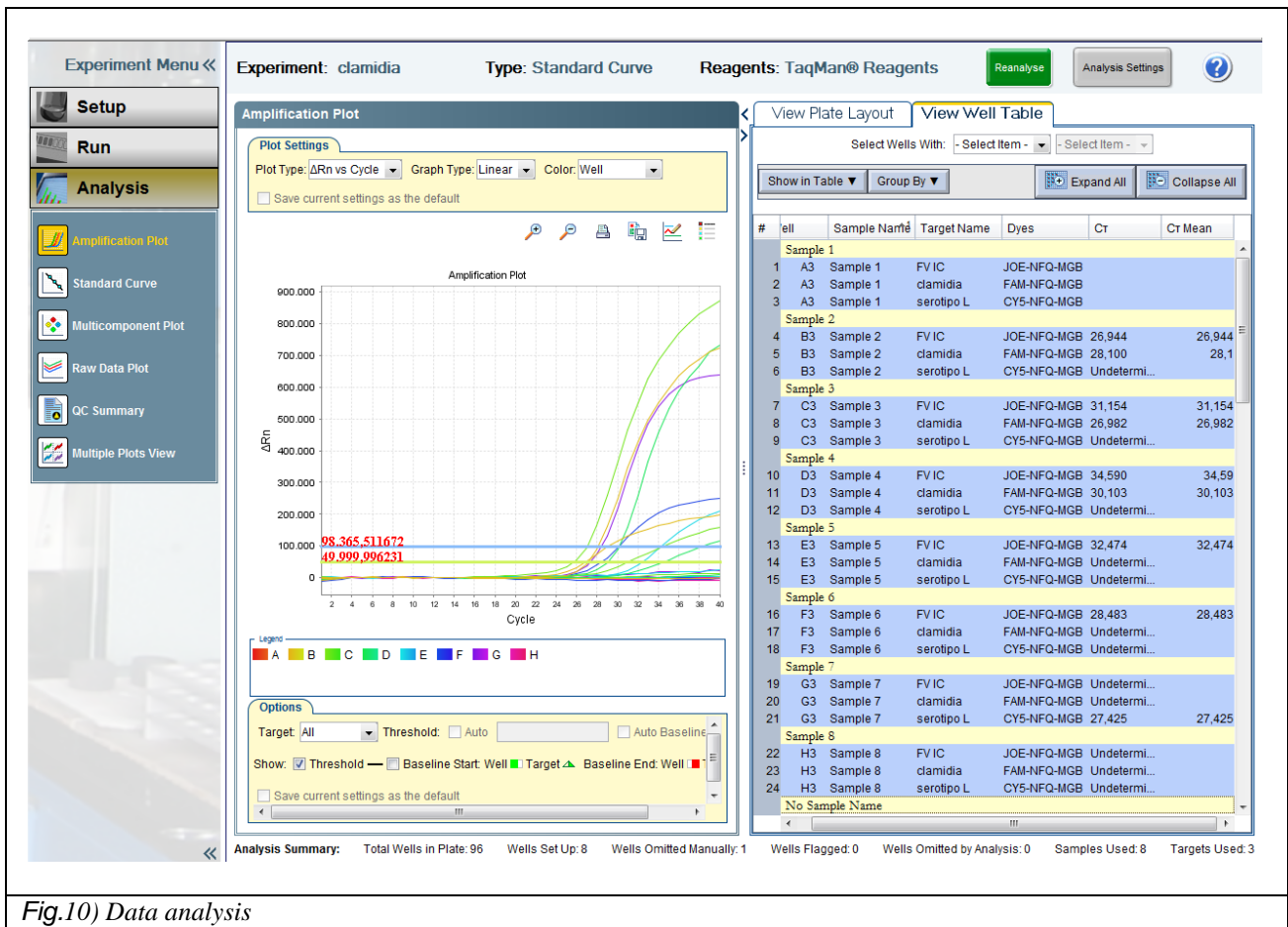
Fig.10) Data analysis

- To generate a report, click "Print Reports" and select the information of interest.

## INTERPRETATION OF RESULTS

For each fluorophore, interpret the results as follows.

NOTE: The signal is considered to be present when the Ct <37. If Ct > 37 it is suggested to analyze the raw data and in case, repeat the sample.

### FAM (*Chlamydia trachomatis*)

Consider the FAM/Green Ct for valid samples and interpret the results as follows:

	FAM/Green Ct	Interpretation
Sample	Present	CT DNA positive
	Absent	CT DNA not detected

### JOE (Factor V –Internal Control)

Consider the JOE/Yellow Ct for valid samples and interpret the results as follows:

	JOE/Yellow Ct	Interpretation
Sample	Present	Valid
	Absent	Invalid*

(\* In the absence of the IC signal in the samples, the result can not be confirmed, then the experiment must be repeated from the extraction. However in the presence of CT DNA at high titer (Ct <20) the absence of the IC signal could be due to competition between the specific target of CT and IC; in this case the positive result for CT can be considered valid.

### Cy5/Red (Serotype L)

Consider the CY5/Red Ct for valid samples and interpret the results as follows:

	FAM/Green Ct	Cy5/Red Ct	Interpretation
Sample	Present	Present	CT DNA positive, serotype L
	Present	Absent	CT DNA positive, serotype A-K
	Absent	Present	Invalid*

(\* Hypothesis of PCR Inhibition or DNA degradation or contamination in some step of the procedure.

## CONTROLS

If in the experiment a negative control (sample negative for CT DNA) and the positive control (supplied in kit) are also analyzed, interpret the results as follows:

	FAM/Green Ct	JOE/Yellow Ct	Cy5/Red Ct	Interpretazione
Negative Control	Absent	Present	Absent	Valid
	Absent	Present	Present	Invalid**
	Absent	Absent	Present/Absent	Invalid*
	Present	Present	Present/Absent	Invalid**
Positive Control	Present	Absent	Present	Valid
	Present	Absent	Absent	Invalid*
	Absent	Absent	Present	Invalid*

(\*Hypothesis PCR Inhibition or DNA degradation in some step of the procedure.

(\*\*) Possible contamination.

## TROUBLESHOOTING

- If both signals (FAM/Green and JOE/Yellow) are not present for a sample, it can be due to PCR inhibition; it is advisable to repeat the assay on that sample.
- In the Positive Control there isn't the Internal Control, therefore JOE/Yellow signal is not expected.
- If in the experiment also a negative control is processed, be sure that it has a positive signal only on JOE/Yellow and does not give any signal of FAM/Green and CY5/Red.
- If in the experiment is present a negative control sample (only mix without DNA), make sure it does not give any signal.
- In case of contamination it is important to find out the source (contamination during the extraction phase, during sample preparation for PCR or contamination from the previous sittings) and repeat the experiment eliminating the possible source of contamination. The use of positive and negative controls can be very useful to ensure proper test performance and to find out contamination origin.

## PERFORMANCE CHARACTERISTICS

### Analytical sensitivity

The analytical sensitivity of the CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit, expressed as the minimum amount of the target marker that can be detected (probability  $\geq 95\%$ ), was determined by testing serial dilutions of a plasmid containing the artificial DNA target (16s gene and cryptic plasmid in the ratio 1:7).

copies	N° of samples	N° of positive	% positive
4800	10	10	100%
480	10	10	100%
240	10	10	100%
160	12	8	66%
120	10	8	80%

The device can detect 245 copies/reaction of DNA target (Probit analysis).

As the kit can detect both 16s gene and the cryptic plasmid (which is present in 7-10 copies for each microorganism), we can conclude that CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit is able to detect till 23-30 organism per reaction.

### ⇒ Diagnostic sensitivity - Sample from routine

The diagnostic sensitivity of the CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit was evaluated by analyzing 141 positive samples. Analysis was performed according to the Instruction for Use (IFU), DNA extraction included.

140 of 141 samples tested were compliant to the expected results, therefore the diagnostic sensitivity of the device is more than 99%.

### ⇒ Diagnostic specificity

The specificity of the CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit was determined by analyzing 165 samples negative for *C. trachomatis*. Analysis was performed according to the IFU, DNA extraction included. No false positive results were obtained. According to these results the specificity of the assay is 100%.

### Potentially cross-reactive markers

To evaluate the potential cross reactivity of CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit with other pathogens, samples from infected patients positive for other diseases (*Mycoplasma genitalium*, *Mycoplasma hominis*, HPV, HCV, HBV, HIV, CMV and *Neisseria gonorrhoeae*) were analyzed. Analysis was performed according to the IFU, DNA extraction included.

Each non-CT sample tested gave a negative result. Therefore CHLAMYDIA TRACHOMATIS REAL TIME 2.0 shows no cross reaction with other pathogens.

### Overall error rate of the system that leads to false-negative

To evaluate the whole system failure rate leading to false negative results, 119 CT negative samples were spiked with the plasmid-mix containing 16s and cryptic plasmid until the 3 x 95% pos. cut-off concentration.

The analysis was performed according to instructions for use, extraction included.

118/119 samples tested gave a positive result, so the false negative rate of CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit is  $< 1\%$ .

## Serotypes

The performance of CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit on different serotypes was determined by analysing 27 CT positive samples of known serotype.

Serotype	N° of samples
E o B/Ba	8
D	1
E	2
I	2
J	2
K	4
L	8

All the samples gave the expected result, so CHLAMYDIA TRACHOMATIS REAL TIME 2.0 kit detection efficiency is independent from the serotype analysed.

## Robustness - Cross contamination

The robustness of the device was determined analysing 68 samples, high positive and negative for *C. trachomatis* in four different work sessions. Analysis was performed according to the IFU, DNA extraction included.

None negative sample gave a positive signal, so CHLAMYDIA TRACHOMATIS REAL TIME 2.0 shows good specificity and no cross-contamination between samples.