BIOLOGIA MOLECOLARE

VER 1 - 22/02/2018

REF AA910/48

48 TEST

CND W0105020307

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HCV RNA REAL TIME QUANTITATIVO 2.0

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BIBLIOGRAFIA

1. Sarrazin C. *Diagnosis of hepatitis C: update 2004. Journal of Gastroenterology and Hepatology (2004) 19, S88-S93*
2. Johnson RJ, Gretch DR, Yamabe H et al. *Membranoproliferative glomerulonephritis associated with hepatitis C virus infection. N Engl J med 1993; 328: 465-470.*
3. Agnello V, Chung RT, Kaplan RM. *A role for hepatitis C virus infection in Type II cryoglobulinemia. N Engl J Med 1992; 327: 1490-1495.*
4. Andreone P, Zignego AL, Cursaro C et al. *Prevalence of monoclonal gammopathies with hepatitis C virus infection . Ann Intern Med 1998; 129: 294-298*
5. Zein NN. *Clinical Significance of Hepatitis C Virus Genotypes. Clinical Microbiology Reviews, Apr. 2000, p. 223-235.*
6. *National Institutes of Health, Consensus Conference Statement. Management of hepatitis C: 2002.*
7. P.T. Hraber et al. *Comparative analysis of hepatitis C virus phylogenies from coding and non-coding regions: the 5' untranslated region (UTR) fails to classify subtypes. Virology Journal 2006, 3:103*
8. M. A. Ansari et al. *HCV-Core Region: Its Significance in HCV-Genotyping and Type Dependent Genomic Expression. Macedonian Journal of Medical Sciences. 2012 Mar 15; 5(1):30-39.*
9. G. Colucci *Molecular diagnostic and predictive tests in the evolution of chronic hepatitis C anti-viral therapies BMC Infect Dis. 2012; 12(Suppl 2): S8. Published online 2012 November 12*

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- *Cq determination mode: single threshold*
- *Baseline Setting: Baseline subtracted curve fit*
- *Apply Fluorescence Drift Correction*
- *Cycles to analyze: from 1 to 45*
- *Baseline threshold: Single Threshold-User defined*-introdurre il valore **25**. Cliccare *OK*

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- *Cq determination mode: single threshold*
- *Baseline Setting: Baseline subtracted curve fit*
- *Cycles to analyze: from 1 to 45*
- *Baseline threshold: Single Threshold-User defined*-introdurre il valore **100**. Cliccare **OK**

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IVD

INTENDED USE

HCV RNA REAL TIME QUANTITATIVE 2.0 test provides reagents for the determination of Hepatitis C viral load (HCV RNA) in human plasma samples by quantitative Real Time RT-PCR of the 5'-untranslated region (5'-UTR) of the viral RNA; the Core region is co-amplified to make the amplicons compatible with NLM HCV genotyping kit GEN-C 2.0 code AC004/24.

A four calibrators panel is included for titre quantification. The Internal Control is endogenous (mRNA of the Housekeeping gene GAPDH) and is co-extracted and co-amplified in order to monitor the complete procedure.

This device has to be used in association with RNA extraction kit NLM code **AA1318** "Viral DNA/RNA extraction" (to be ordered separately).

Viral RNA is retrotranscribed and amplified in one step RT-PCR, using CFX (BioRad).

HCV RNA REAL TIME QUANTITATIVE 2.0 TEST is intended for use in association with clinical presentation and other laboratory markers of HCV infection for the clinical management of patients. The test can be used to assess viral response to antiviral treatment as measured by changes in plasma HCV RNA levels.

The device is not intended for use as a screening test for the presence of HCV RNA in blood products or as a diagnostic test to confirm the presence of HCV infection. The device is for professional use only.

INTRODUCTION

Hepatitis C virus is the main etiologic agent of acute viral hepatitis that can lead, in most patients, to liver cirrhosis and then to hepatocellular carcinoma¹. HCV can also cause extrahepatic diseases²⁻⁴.

HCV genome is a linear single strand RNA with one ORF (Open Reading Frame); in 5' position there is an untranslated highly conserved region (5'-UTR), followed by the Core region, they both represent

the target for viral molecular biology detection. The 5'-UTR and Core genetic heterogeneity among different HCV strains determines the classification into different genotypes^{5,7,8}. HCV has a parenteral transmission through exposure to infected blood. This exposure can occur in the context of blood transfusion before 1992, injection drug use, solid organ transplantation from infected donors, birth from an infected mother, occupational exposure to infected blood, high-risk sexual practice⁶.

After initial exposure, HCV RNA can be detected in patients blood within one to three weeks; acute infection can be severe but rarely is fulminant⁶.

Although generally asymptomatic, about 85% of the acute infections become chronic: persistence of HCV infection is diagnosed by the detection of HCV RNA in the blood for at least six months⁶.

Determination of HCV RNA concentration in plasma samples is an important parameter for management of chronic hepatitis C. While no correlation was observed between HCV RNA absolute viral load and the severity or the progression of liver disease, variation of HCV RNA concentration provides important information about response to treatment in patients undergoing antiviral therapy⁶.

The recent development of directly acting antiviral (DAA)-based regimens and the approval of the first, boceprevir- and telaprevir-based triple-therapy regimes, have brought new hopes for higher virus eradication rate across different disease settings. This calls for even more accurate and sensitive quantitative HCV RNA tests able to monitor faster virus kinetics and promptly detect treatment failures.⁹

With new viral load tests acting as “companion diagnostic”, not only the conventional treatment but also the development and applications of the new generation DAA can be fully supported for a more effective management of chronic hepatitis C and its long-term complications.⁹

REFERENCES

1. Sarrazin C. *Diagnosis of hepatitis C: update 2004*. Journal of Gastroenterology and Hepatology (2004) 19, S88-S93
2. Johnson RJ, Gretch DR, Yamabe H et al. *Membranoproliferative glomerulonephritis associated with hepatitis C virus infection*. N Engl J med 1993; 328: 465-470.
3. Agnello V, Chung RT, Kaplan RM. *A role for hepatitis C virus infection in Type II cryoglobulinemia*. N Engl J Med 1992; 327: 1490-1495.
4. Andreone P, Zignego AL, Cursaro C et al. *Prevalence of monoclonal gammopathies with hepatitis C virus infection*. Ann Intern Med 1998; 129: 294-298
5. Zein NN. *Clinical Significance of Hepatitis C Virus Genotypes*. Clinical Microbiology Reviews, Apr. 2000, p. 223-235.
6. National Institutes of Health, Consensus Conference Statement. *Management of hepatitis C: 2002*.
7. P.T. Hraber et al. *Comparative analysis of hepatitis C virus phylogenies from coding and non-coding regions: the 5' untranslated region (UTR) fails to classify subtypes*. Virology Journal 2006, 3:103
8. M. A. Ansari et al. *HCV-Core Region: Its Significance in HCV-Genotyping and Type Dependent Genomic Expression*. Macedonian Journal of Medical Sciences. 2012 Mar 15; 5(1):30-39.
9. G. Colucci *Molecular diagnostic and predictive tests in the evolution of chronic hepatitis C anti-viral therapies* BMC Infect Dis. 2012; 12(Suppl 2): S8. Published online 2012 November 12

PRINCIPLES OF THE PROCEDURE

HCV RNA REAL TIME QUANTITATIVE 2.0 TEST is based on two processes:

1. Viral RNA extraction
2. Reverse transcription, amplification and detection of target sequence using Real Time RT-PCR

The Internal Control (GAPDH mRNA) is endogenous and therefore extracted together with the sample in order to monitor all the procedure. A four calibrators panel is also included in order to quantify the initial amount of HCV RNA in clinical specimens.

1. Viral RNA extraction

HCV and IC RNA extraction from human plasma samples has to be done using “Viral DNA/RNA extraction” (NLM code AA1318).

2. RT-PCR and Detection

RT-PCR target

The HCV RNA 5'-UTR region has been chosen as target to detect the HCV presence in human plasma samples because it is highly conserved among HCV genotypes.

Co-amplification of the Core region is included, in order to discriminate genotypes 1a ,1b and 6 in case of samples analyzed with NLM AC004/24 Gen-C 2.0 kit.

The internal control is the mRNA of the constitutively expressed GAPDH gene and it is co-extracted and co-amplified together with target HCV; the IC amplicon is similar to HCV target sequence in terms of length and base composition.

Calibrators are obtained by serial dilutions of a synthetic RNA transcript of cloned HCV 5'-UTR and Core regions, therefore they need the same primers and probe as HCV RNA target for Real Time RT-PCR.

Reverse transcription and Amplification

Following extraction step, RNA (HCV and IC) is retrotranscribed and amplified in one-step RT-PCR, using appropriate enzymes and buffer conditions.

Detection

The test is based on the Real-Time PCR, a technique that allows you to monitor in real time the amplification of the samples of interest, using fluorescent probes doubly labeled with a donor fluorophore and a quencher ("dual-labeled probe"). The PCR is carried out in the presence of two specific probes respectively for the internal control and for the 5'-UTR of the virus. The donor fluorophore is different for the two probes: FAM for HCV and JOE for IC. In presence of a light source, when the probes are intact and donor and quencher are close to each other, the fluorescence emitted by the donor fluorophore is absorbed by the proximal quencher. During the amplification each probe matches to its specific sequence and it is target of 5' → 3' nuclease activity of the DNA polymerase. When the donor and acceptor fluorophores are separated, the donor fluorescence can be detected at its specific wavelength: in this way, the amplification of the viral RNA and the IC can be monitored as the reaction proceeds.

CFX (Biorad) comprises in a single instrument a thermal cycler for target amplification and a fluorometer for the detection of fluorescence during the cycling process. The computer connected with the Real Time instrument collects fluorescent data that are displayed in a graph through a specific software (*figure 1*).

Following raw data collection, analysis is carried out. Raw data are normalized to correct the background signal, then a threshold level can be set: this is the level at which fluorescence data are analyzed (*figure2*).

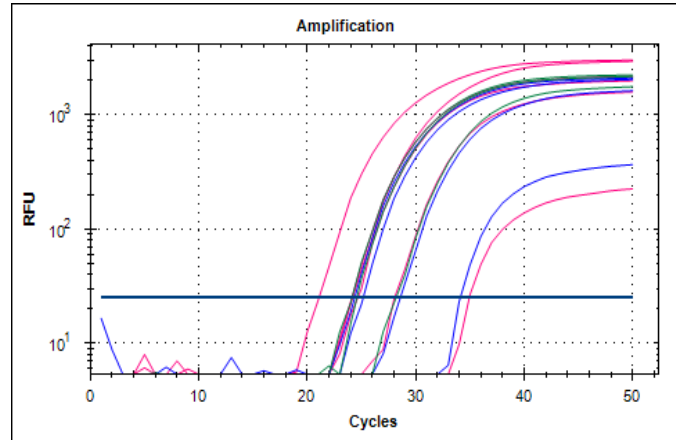
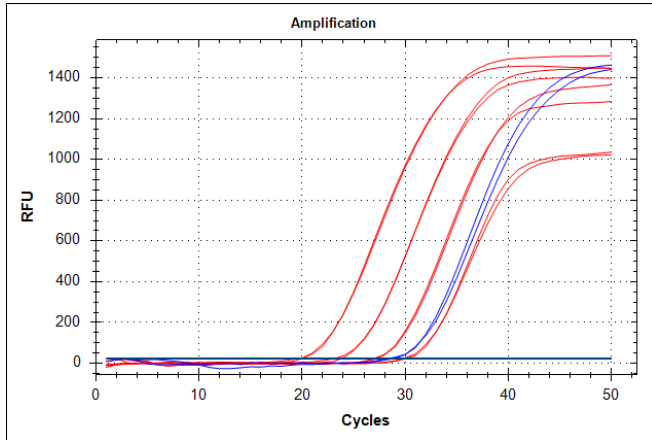
Figure 1: raw data are plotted on a graph of fluorescence versus cycle number.

Figure 2: the normalized data are plotted on a log scale graph of fluorescence versus cycle number.

The number of cycles it takes for a sample to reach the threshold level is named Ct-value (threshold cycle) and it is related to the initial amount of target RNA: the higher is the target titer, the earlier the fluorescent signal reaches the threshold level. If standards with known concentrations are processed, a linear regression analysis produces a standard curve from which the concentration of unknown samples can be determined. The standard curve is generated automatically by plotting the threshold cycle versus standard initial concentration (IU/ml) and calculating the best-fit line; the HCV RNA titre of each sample is determined by locating its Ct on the standard curve (*figure3*).

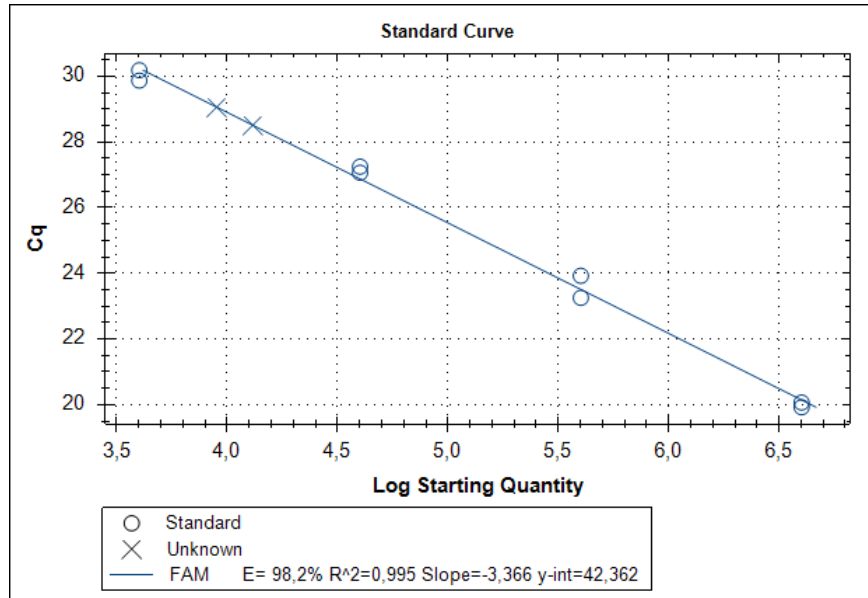
Figure 3: standard curve obtained by linear regression analysis of calibrators Ct plotted versus the respective initial concentration.

REAGENTS COMPOSITION (Store at -25/-15°C)

Mix A 1 x 47 µl

Probes

Primers

Tris-EDTA

Mix B 1 x 35 µl

Primers

Tris-EDTA

Mix C (R36,37,38)  1 x 330 µl

RT-PCR enzymes

MgCl₂

dNTPs

Negative Control 4 x 500 µl

Negative Control is human plasma non reactive for HIV-Ab and RNA, HCV-Ab and RNA, HBS-Ag and DNA.

Negative Control has to be extracted.

Calibrators 1, 2, 3, 4 2 each

HCV Template

Lyophilized calibrators must be resuspended in 14 µl of RNase, DNase and Protease free water before use in RT-PCR. Calibrators concentration can vary in different device lots, therefore it's important to assign to each calibrator the titre reported on the internal label of the package.

Before using calibrators be sure that all the solution is on the bottom of the tube.

MODULARITY

Reagents are provided for 2 distinct runs; 24 samples + 4 calibrators + 1 Negative Control can be analysed in each run. If modularity isn't respected reagents could not be enough for the number of tests declared.

STABILITY AND STORAGE

- All the reagents are stable up to the expiry date indicated on the label when stored at -25/-15°C .
- Thaw all the reagents on ice or at +2/+8°C.
- **Mixes** are stable until expiration date if stored at -25/-15°C . These reagents can be refrozen and thawed max 4 times
- **Mix A** contains FAM and JOE fluorophores that are photosensitive: avoid prolonged exposure to light.
- **Master Mix** must be used immediately after preparation; once dispensed in PCR tubes, the remaining reagent has to be discarded. Avoid prolonged exposure to light.
- **Calibrators** have to be resuspended with RNase, DNase and Protease free water before use; add then the Master Mix in the tubes and transfer the total volume into the plate: it is recommended to dispense them into the plate after RNA samples to avoid cross contamination.

SPECIMEN COLLECTION AND HANDLING

Viral RNA is isolated and purified from human **plasma** specimens prepared within 6 hours from blood collection.

- Collect blood observing universal precautions for venipuncture
- Collect blood in sterile tubes with EDTA as anticoagulant and centrifuge at 1000-1500g for 10-15 minutes to separate plasma. Use only EDTA as an anticoagulant, other types of anticoagulants may interfere with the performance of the test
- Aliquot **450 µl** of plasma for RNA extraction using vertical down-flow airbox and operating to avoid RNA degradation.
- Store samples at +2/+8°C until extraction step. If they are not processed immediately, store them in sterile tubes at -25/-15°C for up to 72 hours prior to freezing at ≤ -70°C.
- Avoid repeated freeze-thaw cycles of plasma samples
- Perform the procedure using universal precautions and handle samples as if capable of transmitting infection.

PRECAUTIONS

- Only professional and opportunely trained personnel should use this kit. Handle this product according to established good laboratory practices and universal precautions throughout the assay procedure.
- All disposable items (tips and tubes) must be DNase, RNase free. Use aerosol-resistant pipette tips to avoid pipettes contamination. Use a new tip every time a volume is dispensed.
- Discard all used material as bio hazardous waste.

- The negative control is a human plasma sample non reactive for HIV-Ab and RNA, HCV-Ab and RNA, HBV HBsAg and DNA. However it should be handled as potentially infectious and should be treated with the necessary safety precautions.
 - Mix C: Irritant (R36, 37, 38); for S phrases refer to MSDS
 - Do not eat, drink, smoke or apply cosmetics in areas where reagents or specimens are handled.
 - If skin or mucous membrane exposure occurs, immediately rinse the area with copious amount of water. Seek medical advice.
 - Do not use device after its expiration date.
 - Do not mix reagents from different lots.
 - Pay attention to device modularity (24 samples + 1 Negative Control + 4 Calibrators for 2 runs): if modularity is not respected the reagents could be insufficient for all the tests declared.
 - Due to the high sensitivity of the test, particular attention should be paid to cross-contaminations. It's recommended to perform the assay in three different areas:
 - o Area 1: pre-PCR (samples handling and extraction)
 - o Area 2: Master Mix preparation.
 - o Area 3: post-PCR (Real Time PCR)
- It's recommended to clean the workstation with chlorine bleach at the end of the procedure.
- Do not use the device if the box is damaged; contact the supplier
 - **It is advisable to have constant and uniform laboratory temperature, avoid to place the instruments near heating/cooling sources that may compromise the right working.**

MATERIAL REQUIRED BUT NOT PROVIDED

AREA 1

Vertical downflow airbox
Dedicated adjustable volume pipettes set and aerosol barrier tips
2 ml and 1,5ml DNase, RNase free tubes
Magnetic separator
Vortex mixer and Heat block
Centrifuge
RNase, DNase and Protease free water

AREA 2

Vertical downflow airbox
Dedicated adjustable volume pipettes set aerosol barrier tips DNase, RNase free
0,2 ml flat cap Real TimePCR, strips or plates DNase, RNase free
RNase, DNase and Protease free water

AREA 3

CFX (BioRad)
CFX (BioRad) software

ASSAY PROCEDURE

VIRAL RNA EXTRACTION

(Extraction method "Viral DNA/RNA Extraction" NLM code AA1318)

Refer to the instructions for use for preparation, handling and disposal of reagents, taking into account 450 µl starting sample quantity.

This protocol is for manual use and involves the use of a 2 ml reaction tubes and a suitable magnetic separator. For the automation of the procedure with the Janus workstation (Perkin Elmer) contact the company.

Prepare the necessary amount of 2 ml centrifuge tubes (samples and negative control). Before starting with the procedure set heat block to +55°C. Ensure that Proteinase K and Poly(A) RNA have been prepared as described.

1. Dispense **in the following order 10 µl Proteinase K** and **450 µl of sample** into 2,0 ml centrifuge tubes. Add **450 µl Lysis Buffer Viral RNA** and then **6 µl Poly(A) RNA**; proceed pipetting up and down for at least **8 times** and incubate at **55°C for 10 minutes**, vortexing ones or twice during incubation.

It is also possible to prepare a premix of Lysis Buffer Viral RNA and Poly(A) RNA, calculating the necessary amount for the number of samples + 1 to process. Use this mix within 15 minutes.

2. Following incubation, briefly centrifuge the samples to avoid contamination when opening the tubes. Add **35 µl** of resuspended **Magnetic Beads Viral DNA/RNA (the Magnetic Bead suspension should be mixed vigorously before dispensing)** and **900 µl of Binding Buffer Viral DNA/RNA** to each tube, mix by pipetting up and down for at least **6 times** and incubate for **5 minutes** at RT.

It is possible to prepare a premix of beads and Binding Buffer Viral DNA/RNA.

3. Place the tubes in a magnetic separator for at least **2 minutes** to let the beads be attracted by the magnets. Remove and discard supernatant by pipetting, **being careful not to take any beads**.
4. Remove the tubes from the magnetic separator. Add **500 µl Wash Buffer A Viral DNA/RNA** to the tube. Resuspend the beads by pipetting until the beads are completely resuspended and incubate for **1 minute** at RT.
5. Place the tube in the magnetic separator to draw the beads to the side of the tube (about 1 minute). Pipette off the supernatant and then remove the tube from the magnet.
6. Repeat the washing procedure (steps 4 and 5) with **500 µl Wash Buffer B Viral DNA/RNA**. **Discard Wash Buffer B Viral DNA/RNA repeating the aspiration with a fine-tipped pipette; be careful to remove all the residual fluid without disturbing the pellet.**
7. Transfer the tubes to the thermoblock and let them air-dry at **55°C** for **20 minutes** to remove any traces of ethanol. The beads-pellet should become light brown when dried, otherwise extend the incubation for further 5 minutes or until completely dry
It is important that the beads are completely dried and that there aren't ethanol drops on the tube walls before continuing with the elution step, as residual ethanol can prevent amplification.
8. Add **70 µl of Elution Buffer Viral DNA/RNA** to the tubes and briefly resuspend the beads by pipetting. Close the tubes, gently vortex the tubes and incubate the suspension for **5 minutes** at **55°C**.
9. Separate the magnetic beads by placing the tubes in the magnetic separator for at least **1 minute**. Transfer the supernatant containing the purified viral RNA to the desired storage tubes.

Purified RNA can be stored at +2/+8°C if immediately used, otherwise keep it frozen at -25/-15°C or at ≤ 70°C for longer periods. It is recommended to thaw RNA at +2/+8°C.

REAL TIME RT-PCR

Set Real Time PCR profile before preparing Master Mix.

Reagents	Volume per sample
Mix A	0,6 μ l
Mix B	0,4 μ l
Mix C	5 μ l

- **Carefully resuspend the mix before use.**
- Prepare the Master Mix for the number of purified samples (clinical specimens and negative control) + 4 calibrators + 2 volumes (to have enough mix for all the samples).
- Mix gently and dispense **6 μ l** of Master Mix in the plate/strip previously marked.
- Add **14 μ l** of purified RNA to each well/tube and mix pipetting up and down.
- At last (to avoid cross contaminations), resuspend the calibrators with **14 μ l** RNase, DNase and Protease free water, then add the mix and transfer all the volume in the specific well/tube .

WARNING: dispense mix and samples very carefully, in order to avoid creating air bubbles. If possible, centrifuge the plate/strip before placing it into the Real Time PCR instrument.

RUN SETUP

Turn on the PC and CFX.

Open the “Bio-Rad CFX Manager Software” and click on “*Create new run*”, select the model of instrument used and click “OK”, the “*Run Setup*” window will open.

It is possible to create a new protocol, select/modify an existing one or repeat a run. To repeat a run with the same settings, open the desired run, click on “*File - Repeat Run*”.

Create a new protocol

- “*Protocol → create New*” (figure 4)
- “*Protocol Editor – new*”
- Enter the reaction volume (**20 µl**) in the “*Sample Volume*” box
- Add the steps by clicking on “*Insert Step*” on the left of the window it is possible to modify temperatures and times directly on the graph or in the test below with a double click):
 1. **50°C for 15 min**
It is possible to modify the temperatures and times on the graph or in the test below with a double click on them.
 2. **95° for 20 sec**
 3. **95°C for 15 sec**
 4. **60°C for 1 min**; add “*plate read*” in this step
 5. **Insert GOTO 3, 44 more times**
 6. **Optional: for downstream application with GEN-C 2.0 (cod. NLM AC004/24), add 10°C forever (type 00:00 and press Enter)**
- Click OK. Save the new protocol.

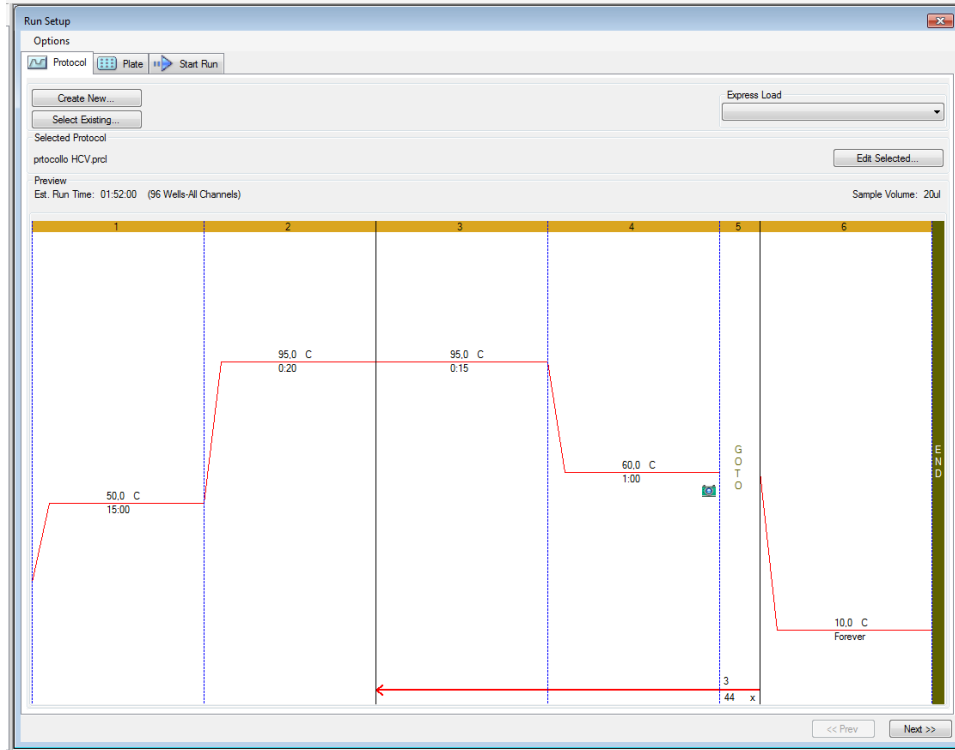
Figure 4

PLATE SETUP

In “*Run setup*” window edit “*plate*” editor window. It is possible to create a new plate or select/modify an existing one.

Create a new plate

- “*Plate* → *create New*”
- In “*Plate Editor – new*” window (*figure 5*) in “*Scan Mode*” select “*All channels*”
- Click “*Select Fluorophores*” button and check FAM and JOE
- Select a well and on the right side of the window Choose “*Sample Type* → *Unknown, Standard, Positive or Negative Control*” depending on the type of sample
- Type *Target Name* (*HCV* for FAM and *Internal Control* for JOE) and select *Load* check boxes to insert fluorophores (FAM and JOE for each well except for the standards and where only FAM has to be loaded).
- Type “*Sample Name*” and click “*Load*”. Check boxes to insert sample Name
- For the *Standards*, insert the right concentration values as indicated on the label inside the box of the device and click the load button in the concentration window.
- Click OK. Save the new plate.

Select/modify existing protocol or plate

- To import an existing protocol or plate click “*Select Existing*” in protocol or plate window.
- To modify an existing plate click “*Select Existing*” → “*Edit Select*”.
- Proceed as explained above.

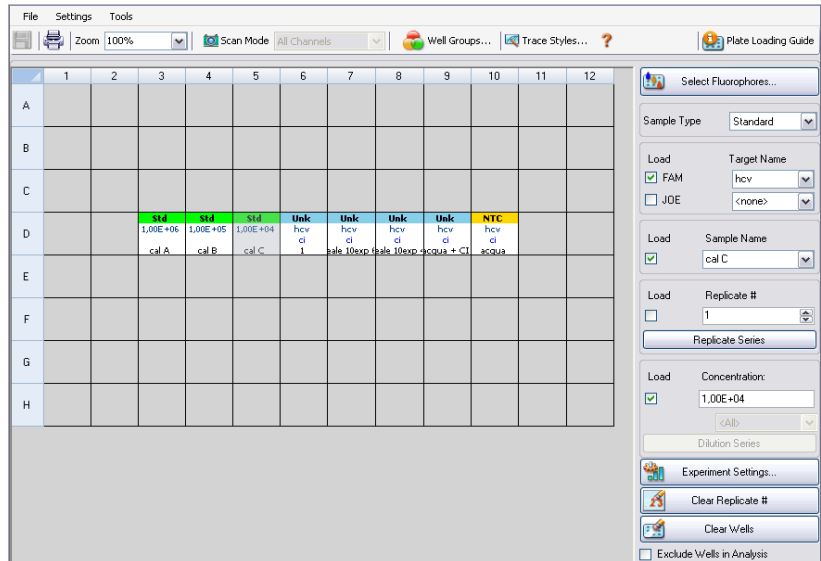
Figure 5

START RUN

- Prepare the plate/strip with samples and mix.
- In “*Start Run*” tab click –“*Open Lid*”, place the plate/strip in CFX instrument and click“*Close Lid*”
- Press “*Start Run*” to begin.
- It’s possible to monitor the Raw data plots during the assay in “*Run Details*” window,in “*Real Time status*” page.

DATA ANALYSIS

At the end of the assay click “Stop” to exit from *10°C forever* mode. The window “*Data Analysis*” will open automatically.

To exclude a sample/calibrator from the analysis go to “*plate setup-view/edit plate*” (in the upper right). Click on the well to omit and then on “*Exclude well in analysis*” (in the lower right).

Analyze data for FAM and JOE separately.

a) **FAM:** deselect Joe in the amplification window; click on “*Settings*” and select (figure 6):

- *Cq determination mode: single treshold*
- *Baseline Setting: Baseline subtracted curve fit*
- *Apply Fluorescence Drift Correction*
- *Cycles to analyze: from 1 to 45*
- *Baseline threshold: Single Threshold-User defined-introduce the value **25**. Click OK*

In “*Quantification*” for the data analysis: the graph on the left shows the amplification data (linear or log scale), the graph on the right shows the standard curve. The R^2 value should be as close as possible to 1. If one of the four calibrators doesn’t match with the linear fitting (i.e. in the presence of a Ct value outside the corresponding acceptability RANGE reported on the label inside the box), it can be excluded from analysis and a more precise standard curve is plotted; this allows a more precise sample quantification without altering the clinical significance of the results.

If more than one calibrator doesn’t match with the linear fitting, Standard Curve cannot be plotted and the run must be repeated.

Check FAM results for valid samples: a value of concentration (IU/ml) is given in the result table.

- If given concentration is lower than Detection Limit, report result as HCV RNA < 18 IU/ml; in this case positive result reproducibility is not guaranteed.

- If sample has signal in the Joe channel but no signal in the FAM channel report as *Target not detected*
- Results that are not within the linear range of the assay cannot be considered accurate, therefore they should be interpreted with caution.
- If given concentration is above the upper quantification limit, report result as HCV RNA > 2,00E+08 IU/ml. To get a more accurate result, the test should be repeated using the starting plasma sample diluted with negative plasma. The calculated concentration must be multiplied by the dilution factor to obtain the initial concentration of that sample.

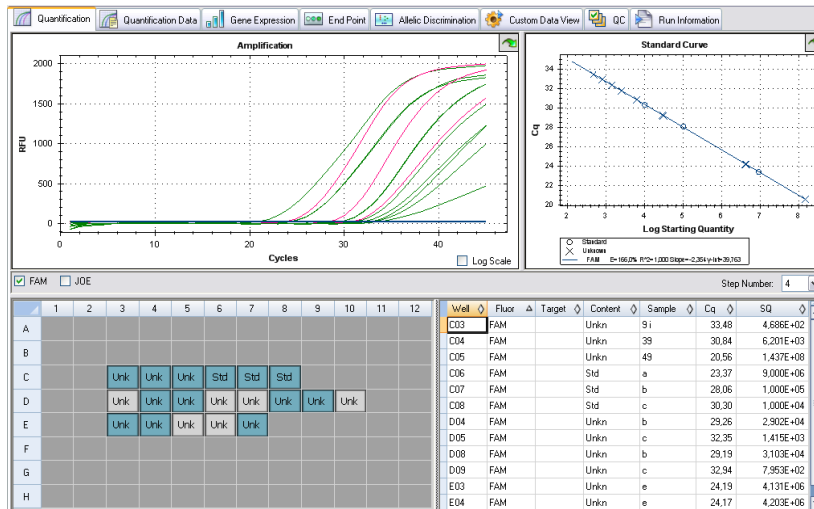
Figure 6

b) **JOE**: deselect FAM in the amplification window. Click on “*Settings*” and select

- *Cq determination mode: single threshold*
- *Baseline Setting: Baseline subtracted curve fit*
- *Cycles to analyze: from 1 to 45*
- *Baseline threshold: Single Threshold-User defined-introduce the value 100. Click OK*
- Check samples JOE Ct and interpret results as follows:

	JOE Ct	Interpretation
Sample	Present	Valid
	Absent	Not Valid*

(*) In absence of IC signal sample result can't be confirmed, therefore sample run should be repeated.

- If negative controls have been tested, interpret results as follows:

	FAM Ct	JOE Ct	Interpretation
Negative Control	Absent	Present	Valid
	Absent	Absent	Not Valid*
	Present	Present	Not Valid**

(*) Hypothesis of RT-PCR inhibition or RNA degradation in one of the procedure steps

(**) Possible contamination

Create a Report (“*Tools* → *Reports*”). Select/Deselect useful information.

WARNING: for a correct interpretation of results, always pay attention to the graph of the session.

Example 1 (figure 7a): the sample highlighted in red in the graph shows a positive Ct value (35), but the graph itself shows it is an instrumental artifact. The found Ct value should not be taken into account and the sample must be repeated.

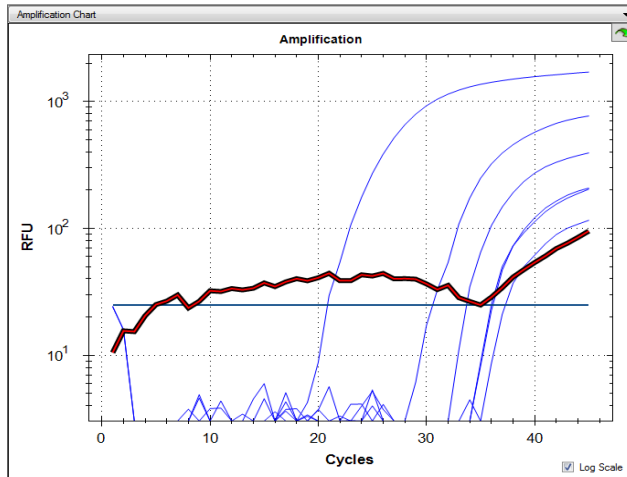
Figure 7a

The table is titled 'Quantification Summary Results Grid'. A yellow arrow points to the row for well C03. The table has columns for Well, Fluor, Target, Content, Sample, Cq, and SQ.

Well	Fluor	Target	Content	Sample	Cq	SQ
C03	FAM		Unkn		35.03	5.971E+01
C06	FAM		Unkn		36.00	3.097E+01
C07	FAM		Unkn		37.25	1.332E+01
C08	FAM		Std-1		36.10	8.000E+01
C09	FAM		Std-1		33.59	8.000E+01
C10	FAM		Std		30.51	8.000E+02
E03	FAM		Unkn		N/A	N/A
E04	FAM		Std		20.78	1.000E+06

Example 2 (figure 7b I and II): If fluorescence signals at the beginning of the run are dishomogeneous, it is possible to exclude the first cycles; select from “Settings” - Cycles to analyze: from 10 to 45

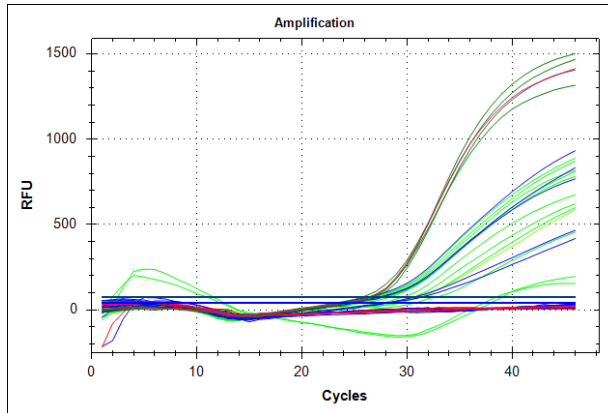
Figure 7b I

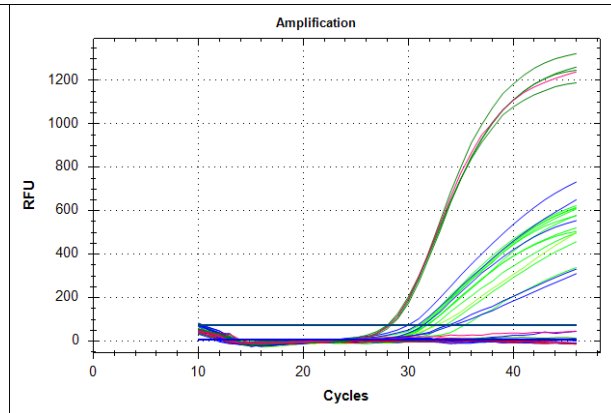
Figure 7b II

WARNING

- In case of absence of expected JOE signal it is advisable to repeat the assay.
- Calibrators haven't Internal Control, therefore JOE signal is not expected.
- When contamination events occur it's important to find out the source (extraction and/or RT-PCR steps). Controls provided can be very useful to ensure the correct test performance and to find out contamination origin.

PERFORMANCE CHARACTERISTICS

Specificity

The specificity of the HCV RNA REAL TIME QUANTITATIVE 2.0 TEST was determined by analysing 501 HCV negative plasma samples from blood donors. No false positive results were obtained. According to the results the specificity of the assay is 100%.

Analytical sensitivity

Sensitivity was determined by analysing four dilution levels of an HCV positive clinical specimen calibrated with the HCV WHO 3rd International Standard (NIBSC code 06/100).

For the dilutions a plasma negative for HCV RNA and antibodies was used. The analysis was performed according to instructions for use, RNA extraction included. As shown in table below, the detection limit of HCV RNA REAL TIME QUANTITATIVE 2.0 TEST is **18 IU/ml** (Probit analysis at 95%).

Dilutions	N tot. runs	N tot. positives	% positives
A (100 IU/ml)	24	24	100%
B (50 IU/ml)	24	24	100%
C (25 IU/ml)	24	23	96%
D (12,5 IU/ml)	24	22	92%

*Probit Analysis done with Statplus 2009

Linear Range

To evaluate the linear range of the device 8 dilution levels were prepared using a synthetic HCV RNA diluted in negative human plasma. The use of a synthetic HCV RNA (calibrated with the 3rd WHO International Standard NIBSC 06/100) is due to the unavailability of a clinical sample with a very high viral load. The test was performed according to the instructions for use using three different lots of reagents. Each dilution level has been tested in 15-29 replicates.

Diluizione	Titolo (UI/ml)
A	2,00E+08
B	2,00E+07
C	2,00E+06
D	2,00E+05
E	2,00E+04
F	2,00E+03
G	2,00E+02
H	4,00E+01

As shown in figure below (*figure 8*), HCV RNA REAL TIME QUANTITATIVE 2.0 showed a linear response from 4,00E+01 IU/ml to at least 2,00E+08 IU/ml, using the accuracy acceptance criterion of: $\pm 0,3 \log 10$.

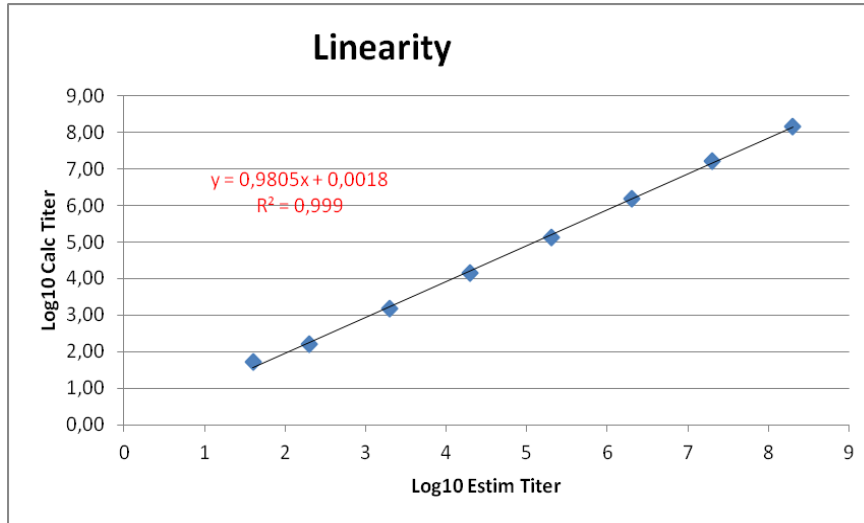
Figure 8

Precision

Intra-assay precision

Intra-assay precision was evaluated by using dilution levels prepared as described for Linear Range determination.

Two different operators tested 3 replicates per dilution level using the same batch lot.

Inter-assay precision

Inter-assay precision was evaluated by using the same samples as the intra-assay precision.

Two different operators tested several replicates per dilution level in different runs and days, using three different lots of reagents.

The test was performed according to the instructions for use, starting from the RNA extraction; in this way the precision was evaluated for all the steps of the procedure.

Results of intra-assay and inter-assay tests are showed in the table below.

Conc (IU/ml)	2,00E+07	2,00E+07	2,00E+06	2,00E+05	2,00E+04	2,00E+03
Standard Dev. Ct	0,199	0,316	0,282	0,214	0,181	0,238

HCV Genotypes

The performance of HCV RNA REAL TIME QUANTITATIVE 2.0 TEST on different HCV genotypes was determined by analysing four dilution levels of different HCV positive clinical specimens genotypes (as listed below). The viral load of the samples used was determined by comparing the signal of the original samples with the signal of the HCV WHO 3rd International Standard (NIBSC code 06/100).

Genotypes tested: 1a, 1b, 2a, 2b, 2a/2c, 3a, 4c/4d, 5 and 6 (determined with NLM GEN C cod AC004 and sequencing).

Dilutions analyzed:

Gen 1a: da $1,5 \times 10^2$ a $2,8 \times 10^6$ IU/ml

Gen 1b: da $1,1 \times 10^2$ a $4,3 \times 10^6$ IU/ml

Gen 2: da $9,8 \times 10^1$ a $2,7 \times 10^6$ IU/ml

Gen 3: da $1,5 \times 10^2$ a $2,8 \times 10^6$ IU/ml

Gen 4: da $4,5 \times 10^2$ a $1,8 \times 10^6$ IU/ml

Gen 5: 3×10^2

Gen 6: 7×10^2

As showed in the graphic below (*figure 9*) quantification of HCV RNA is comparable for the different genotypes analysed. Every tested samples gave a positive result, as expected, therefore quantification of HCV RNA REAL TIME QUANTITATIVE 2.0 TEST is independent from the genotype.

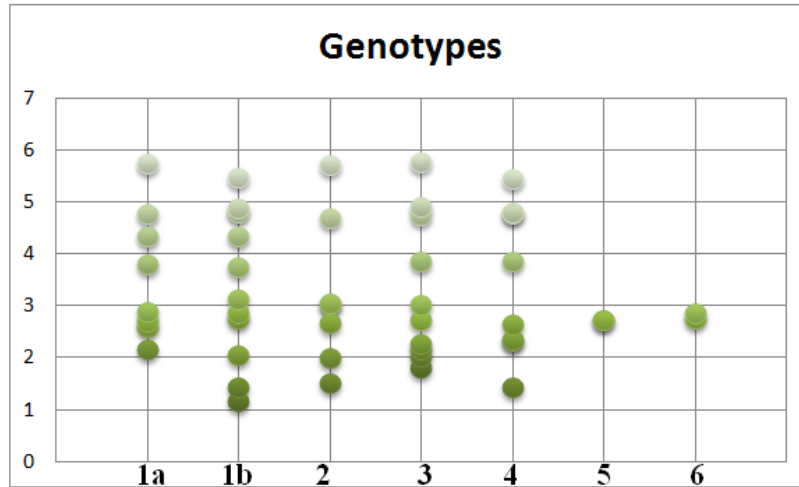
Figure 9: Samples genotype was determined using NLM GEN-C cod. AC004

Potential cross reactive markers

To evaluate the potential cross reactivity of HCV RNA REAL TIME QUANTITATIVE 2.0 TEST with other pathogens the following samples (from infected patients) were analysed:

Other pathogens	Nr of samples tested
HGV	2
HCMV	2
HIV	2
HBV	2
HPV	2
Herpes Simplex Virus	2
Chlamydia trachomatis	2

Each non-HCV samples tested gave a negative result. Therefore HCV RNA REAL TIME QUANTITATIVE 2.0 TEST showed no cross reaction with other pathogens.

Potentially interfering substances

Data not available.

Comparsion with ABBOTT REAL TIME HCV (ABBOTT)

The performance of the HCV RNA REAL TIME QUANTITATIVE 2.0 TEST was compared to Abbott Real Time HCV (Abbott) analysing 102 HCV samples.

The test was performed according to the instructions for use, including the extraction phase. Results obtained show a good correlation between these two assays (*figure 10*).

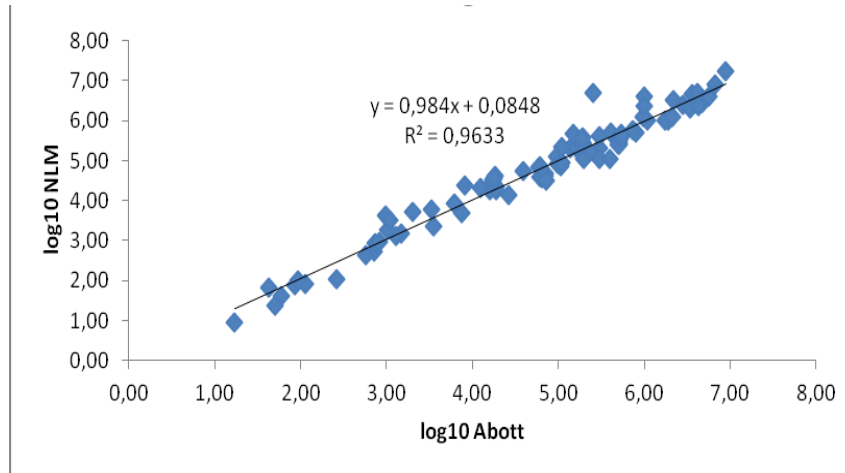
Figure 10

Device performances with positive samples reflecting routine conditions of users

To evaluate device performances with samples reflecting routine conditions of users, 105 plasma specimens positive for HCV were tested with HCV RNA REAL TIME QUANTITATIVE 2.0 TEST and gave a result as expected.

The samples were from patients with a first diagnosis of HCV or in follow up before, during or at the end of pharmacological therapy.

The same samples were quantified with Abbott RealTime HCV assay as reference system.